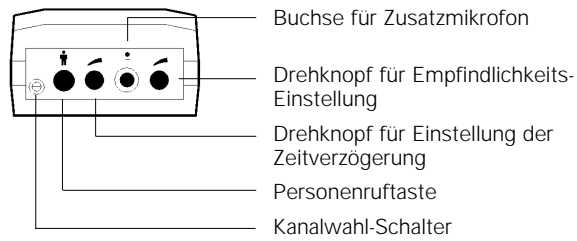
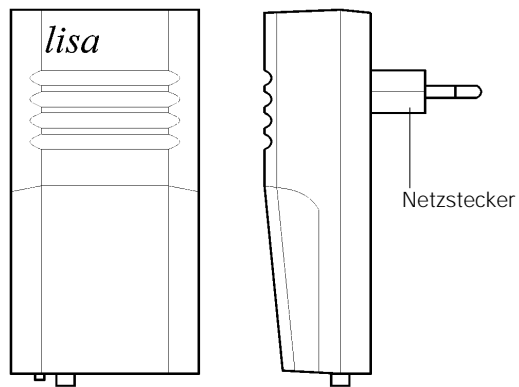
Babysender
Baby cry transmitter
Emetteur cris de bébé
Babyzender
Trasmittitore baby
Emisor de gritos de bebé

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Bedienungsanleitung
Operating Instructions
Mode d'emploi
Gebruiksaanwijzing
Istruzioni per l'uso
Instrucciones de servicio

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
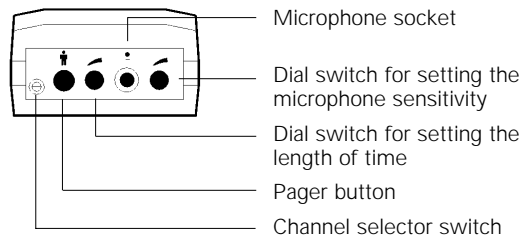
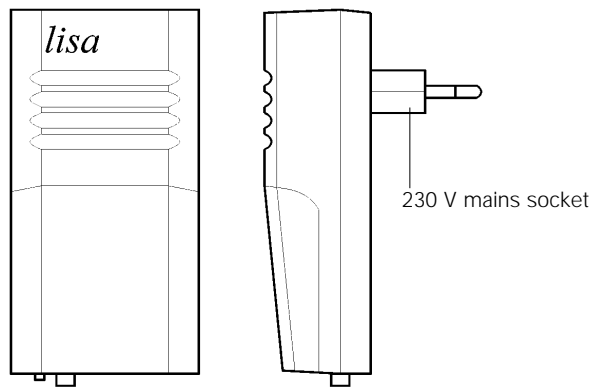
- Microphone socket
- Dial switch for setting the microphone sensitivity
- Dial switch for setting the length of time
- Pager button
- Channel selector switch

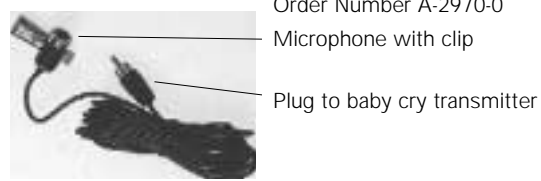Microphone cable 4 m
Order Number A-2970-0

- Microphone with clip
- Plug to baby cry transmitter

Congratulations on purchasing the *Baby Cry Transmitter*. We hope it will give you many years of reassuring operation. Please read through these operating instructions carefully. They tell you how to set up the system correctly and explain all the system's features to you.

lisa system introduction

An alerting system requires at least one transmitter and a receiver. The transmitter captures sound signals and transforms them into digital signals that are passed to a receiver by the existing mains electricity supply. The receiver then transforms these digital signals into light signals.

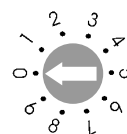
Channel selection

A digital transmission system ensures the correct recognition of each of the digital signals from each transmitter. The system has 10 different selectable channels, 0-9. The factory setting for all *lisa* devices is channel 0. Only where more than one installation is required in the same building will it be necessary to select a different channel for each system.

To select a channel, turn the rotary switch on the underside

of the transmitter to the required channel number using a small screwdriver.

Example:
Set channel number 0

Personal Call button

Pressing this button causes a certain flash sequence on each receiver. It is possible to call people in other rooms this way. This button also allows the *lisa* system to be tested without any connecting cables.

Duration of flashing signals

The factory setting for the duration of the flash is 28 seconds for all *lisa* receivers. However, you can reduce this to 8 seconds if you wish. To change the flash duration for a particular transmitter, remove the transmitter from the mains socket and, while pressing in the Personal Call button, plug the transmitter back into the socket. To switch back to 28 seconds remove the transmitter from the socket and plug back in again this time without pressing the Personal Call button.

Baby Cry Transmitter operation

The *Baby Cry Transmitter* contains a built-in microphone to pick-up the sound of a crying baby. Plug the transmitter into a convenient mains socket near to the baby.

On the underside of the transmitter are two knobs.

The knob labelled **S** adjusts the microphone sensitivity.

The knob labelled **T** adjusts the length of time the sound must be on before the transmitter activates.

Knob **T** in position 0 (fully anti-clockwise) = minimum respond time (approx. 0.2 sec.)

Knob **T** in position 3 (fully clockwise) = maximum respond time (ca. 3 sec.)

Knob **S** in position 0 (fully anti-clockwise) = minimum microphone sensitivity

Knob **S** in position 3 (fully clockwise) = maximum microphone sensitivity.

An initial setting with high microphone sensitivity and average response time (approx. 2.5 sec.) is recommended.

Knob **S** = 3 (fully clockwise)

Knob **T** = 1.5 (half way round)

With this setting the transmitter does not react to short loud sounds, but reacts to sounds with a constant sound level that are uninterrupted for approx. 2.5 seconds (e.g. baby cry). However, you will need to experiment with these settings to find the most convenient settings.

Additional 4-metre lead microphone (accessory):

Simply plug the microphone cable into the socket on the underside of the baby cry transmitter. The microphone can now be placed very near to the baby if the built-in microphone is too far from the baby to pick up the sound adequately.

Warning

The additional microphone and lead could prove a danger to a baby or small child. Ensure that they are well out of reach.

Important note

All *lisa* devices connected to the mains supply have a low power consumption which may lead to the device feeling warm. This is normal and does not interfere with the normal function of the devices.

These signalling systems have a high operational safety.

Should any faults occur in spite of proper installation and use, please contact your dealer.

High and low voltage spikes (e.g. during a thunderstorm) can occur on the mains supply. The *lisa* devices have an integrated electronic controller which may stop working after these conditions. If this happens simply unplug each of the devices and plug them in again.

As a precaution remove all devices from their sockets during a thunderstorm.

The use of halogen lamps of or other equipment (e.g. Microwave Ovens) may, in rare cases, interfere with the *lisa* signals. In these cases the use of mains filters or suppressors is advisable. Please ask your dealer.

Maintenance and care

All *lisa* devices are maintenance free. To clean, remove from their mains sockets and wipe over with a moistened cloth. Never use alcohol, thinner or other organic materials.

lisa devices should not be exposed to excessive direct sunlight, heat, thick dust, humidity or mechanical shock.

Warranty

This warranty covers the repair of the product and returning it to you free of charge. It is essential that you send in the product in its original packaging, so do not throw the packaging away.

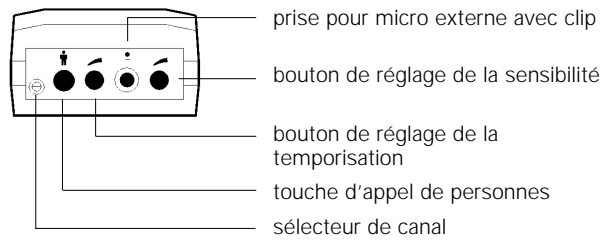
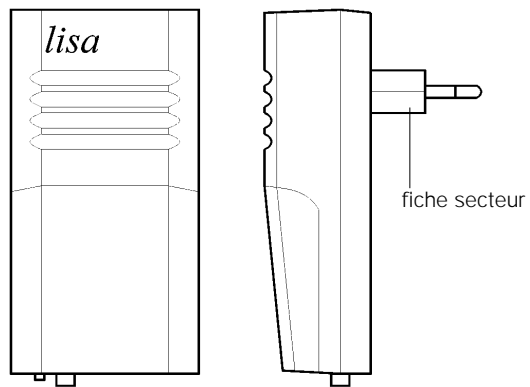
The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people who are not authorised to do so (destruction of the seal on the unit). Repairs under warranty are only carried out providing you have filled in and returned the enclosed warranty card from the dealer.

Technical data

Power supply:
230 V, 50 Hz
Power consumption:
approx. 1 W

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Humantechnik GHL AG
ELECTRONIQUE S.A.R.L