HUMANTECHNIK

*lisa*

D

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Seite 2

combi II *Sender*

GB

**Operating Instructions**

Page 8

combi II *transmitter*

F

**Mode d'emploi**

Page 12

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NL

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Pagina 18

combi II *Zender*

I

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Pagina 22

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

E

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Página 28

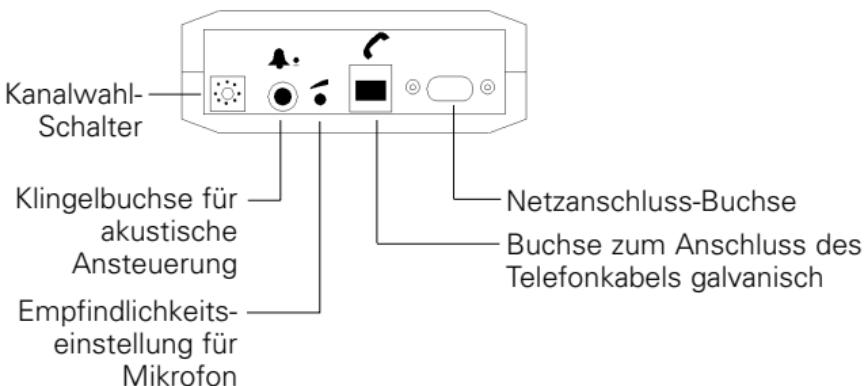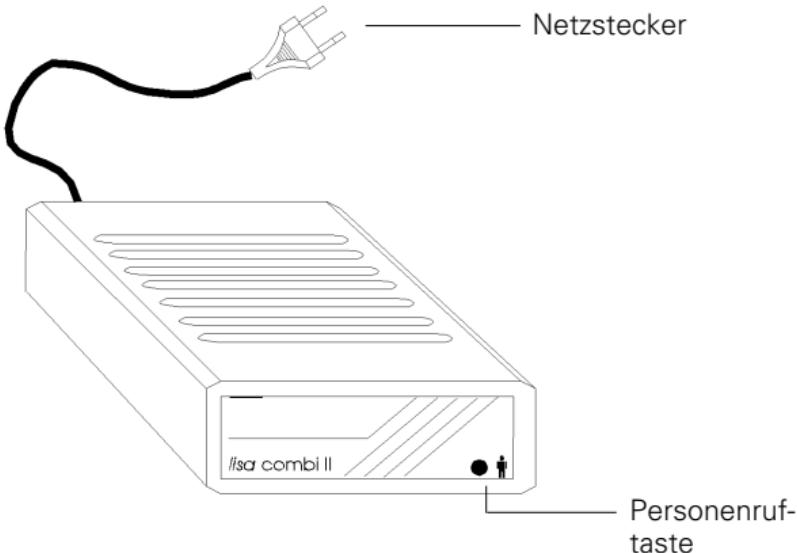combi II *Emisor*

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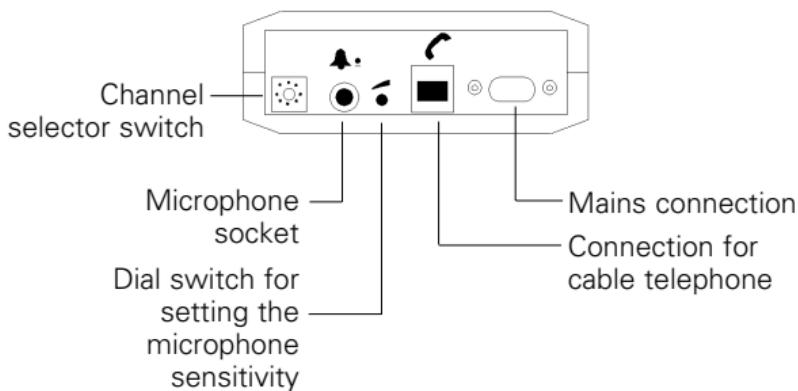
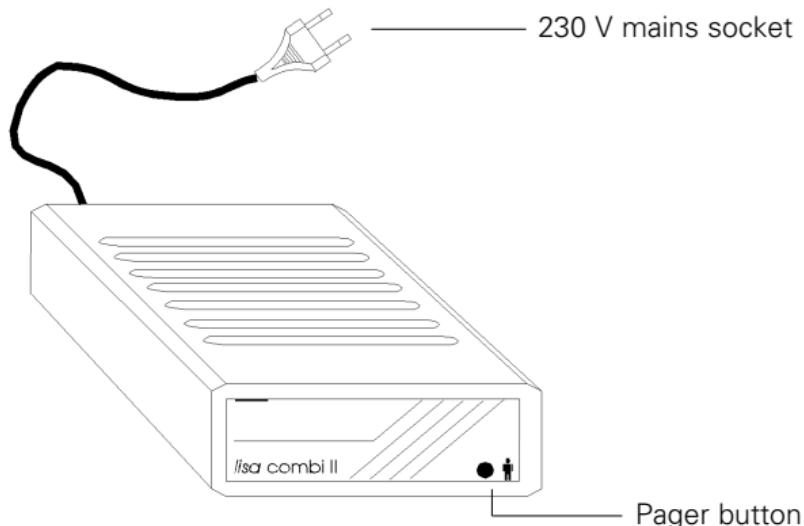
A-2951-0 (length 10 m)

Plug to connection to combi II

Microphone to stick over the acoustic output of the doorbell.

Congratulations on purchasing the *lisa Combi II*. We hope it will give you many years of reassuring operation. Please read through these operating instructions carefully. They tell you how to set up the system correctly and explain all the system's features to you.

### ***lisa* system introduction**

An alerting system requires at least one transmitter and a receiver. The transmitter captures sound signals and transforms them into digital signals that are passed to a receiver by the existing mains electricity supply. The receiver then transforms these digital signals into light signals.

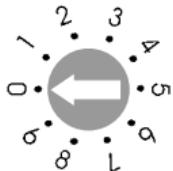
### **Channel selection**

A digital transmission system ensures the correct recognition of each of the digital signals from each transmitter. The system has 10 different selectable channels, 0-9. The factory setting for all *lisa* devices is channel 0. Only where more than one installation is required in the same building will it be necessary to select a different channel for each system.

To select a channel, turn the rotary switch on the underside of the transmitter to the required channel number using a small screwdriver.

### **Example:**

Set channel number 0

### **Personal Call button**

Pressing this button causes a certain flash sequence on each receiver. It is possible to call people in other rooms this way. This button also allows the *lisa* system to be tested without any connecting cables.

### **Duration of flashing signals**

The factory setting for the duration of the flash is 28 seconds for all *lisa* receivers. However, you can reduce this to 8 seconds if you wish.

To change the flash duration for a particular transmitter, remove the transmitter from the mains socket and, while pressing in the Personal Call button, plug the transmitter back into the socket. To switch back to 28 seconds remove the transmitter from the socket and plug back in again this time without pressing the Personal Call button.

### **Doorbell connection**

Plug the transmitter into a convenient mains socket. Plug the doorbell microphone lead into the respective socket at the back of

the transmitter. Lay the cable to the door chime and attach the microphone to the door chime by peeling off the paper circle to reveal the sticky ring beneath. For optimal operation of the transmitter the sensitivity may be adjusted so that only the doorbell activates the transmitter and not background noise.

### **Setting the sensitivity**

1. Set the sensitivity controller to its lowest level (turn to the anti-clockwise to stop)
2. Activate the doorbell and gradually increase the sensitivity of the transmitter (by turning the dial clockwise) until the transmitter reacts to the doorbell.
3. If your intercom system operates with different signal tones, make sure that both signal tones are reliably detected.

### **Telephone connection**

Plug the transmitter into a convenient mains socket. Plug the end of the cable with the transparent clip-in connector (6-pin plug) into the socket at the underside of the transmitter. Plug the other end into the telephone socket.

If the required telephone socket is in use plug a 3-way telephone adaptor into the socket and plug your telephone into one of the sockets on the adaptor.

### **Warning**

Only use the supplied telephone cable otherwise correct operation can not be guaranteed. The *lisa Combi II* is only designed for connection to analogue systems. If a connection is required to a digital system (e.g. ISDN or part of a PBX exchange system) please use a Telephone Transmitter- Acoustic or the *Universal lisa*.

### **Important note**

All *lisa* devices connected to the mains supply have a low power consumption which may lead to the device feeling warm. This is normal and does not interfere with the normal function of the devices. These signalling systems have a high operational safety. Should any faults occur in spite of proper installation and use, please contact your dealer.

High and low voltage spikes (e.g. during a thunderstorm) can occur on the mains supply. The *lisa* devices have an integrated electronic controller which may stop working after these conditions. If this happens simply unplug each of the devices and plug them in again.

**As a precaution remove all devices from their sockets during a thunderstorm.**

The use of halogen lamps of or other equipment (e.g. Microwave

Ovens) may, in rare cases, interfere with the *lisa* signals. In these cases the use of mains filters or suppressors is advisable. Please ask your dealer.

### **Maintenance and care**

All *lisa* devices are maintenance free. To clean, remove from their mains sockets and wipe over with a moistened cloth. Never use alcohol, thinner or other organic materials.

*lisa* devices should not be exposed to excessive direct sunlight, heat, thick dust, humidity or mechanical shock.

### **Warranty**

This warranty covers the repair of the product and returning it to you free of charge. It is essential that you send in the product in its original packaging, so do not throw the packaging away.

The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people who are not authorised to do so (destruction of the seal on the unit).

Repairs under warranty are only carried out providing you have filled in and returned the enclosed warranty card from the dealer.

### **Disposal**

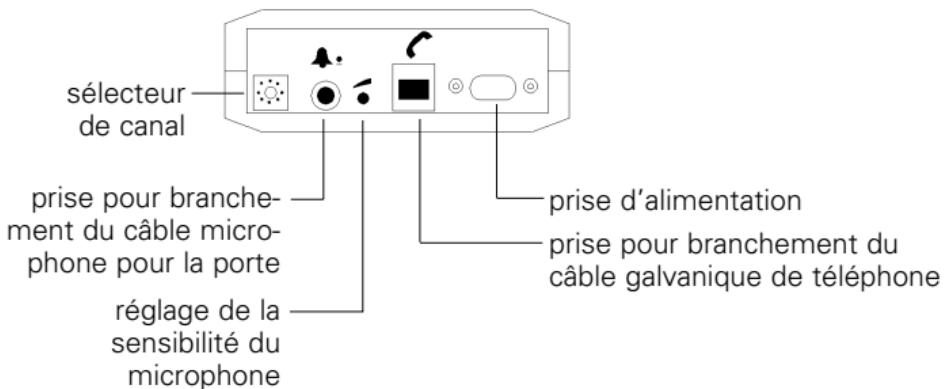
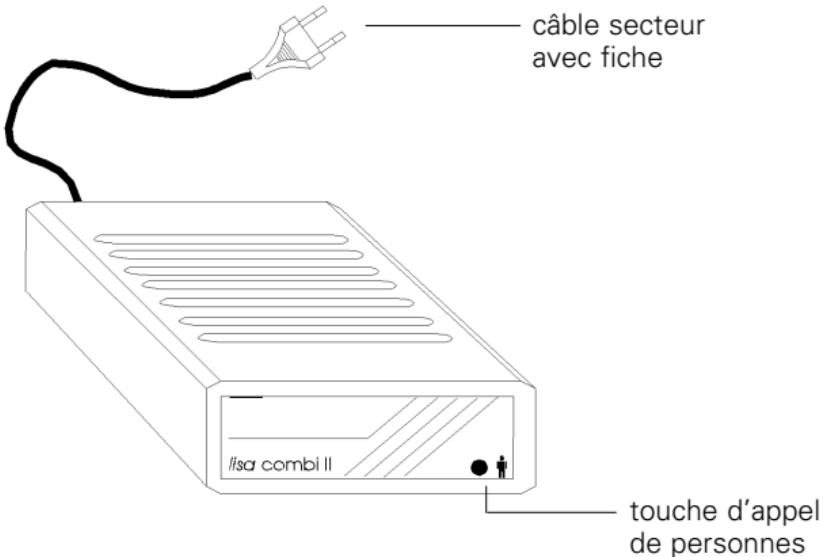
— Disposal of used electric and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system). The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electric and electronic units. You protect the environment and health of your fellow men by the correct disposal of this products. Environment and health are endangered by a faulty disposal. Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of this product from your local community, your communal disposal company or your local dealer.

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**Humanteknik GHL AG**