HUMANTECHNIK

***Crescendo 50***

**D**

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Seite 2

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**GB**

**Operating Instructions**

Page 8

Personal Listener

**F**

**Mode d'emploi**

Page 14

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**NL**

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Pagina 20

Hoorversterker

**I**

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Pagina 26

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**E**

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Página 32

Amplificador audiométrico

## Bedienelemente

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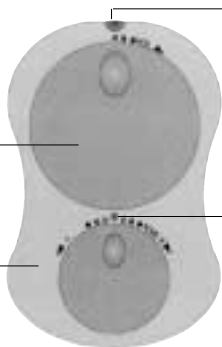
Lautstärke-  
regler

Klang-  
regler

Internes  
Mikrofon

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
Batteriefach

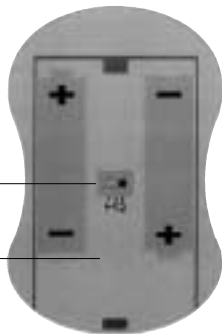Batteriefach-Deckel

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
Internes Mikrofon

Mikrofon-Buchse

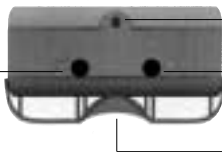Anhängeclip

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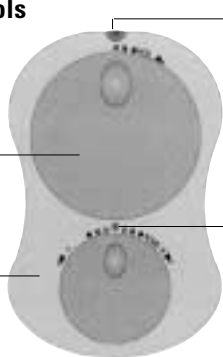
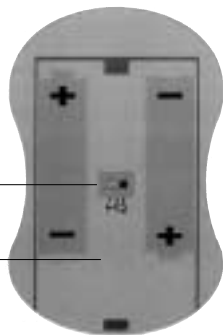
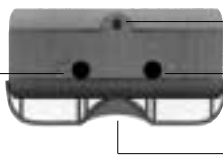
Clip

## Introduction

Congratulations on purchasing your *Crescendo 50 amplifier*. You have selected a modern and reliable system. Please read the operating instructions thoroughly before use in order to familiarise yourself with the amplifier and benefit from all the capabilities of the system.

The *Crescendo 50* receives sound from either the built-in or external microphone and amplifies it. The amplifier can also receive sound from the output of other audio equipment. The amplifier is equipped with a pick up coil which enables you to receive signals from an inductive loop system.

## Contents

The following items are enclosed:

- *Crescendo 50 amplifier* with internal microphone
- 2 x 1,5 V AA batteries
- Headset with stereo plug
- Belt Clip
- Neck cord
- Operating instructions
- Warranty card

## Inserting batteries

The amplifier is operated with 2 AA batteries (Type LR6 or AA).

The battery compartment is located at the back of the amplifier.

To insert the batteries:

1. Open the battery compartment by pressing in the clip at the end of the unit and lift the cover off.
2. Insert the two batteries according to the polarity marks.
3. Close the battery compartment.

## Listening

Sound can be received by the amplifier in three different ways according to the Operating Mode – **M**, **A** or **T**.

- M** Sound is received through a microphone, built-in or external.
- A** Auxiliary or direct connection to TV, HiFi or other sound source.
- T** "T" / Inductive Loop system listening.

## Operating

Select the operating mode with the amplifier switched off.

- Open the battery cover and adjust the switch to the required setting, replace the battery cover.
- To switch the amplifier on turn the volume control clockwise past the "click". A green LED will light up to indicate the amplifier is on.
- Adjust the volume to suit your listening level and the tone control to boost the high or low tones.
- To switch the amplifier off turn the volume control anti-clockwise until it "clicks". The green LED is turned off.

## Problems

### No sound

- Check Headset/Microphone are in correct sockets.
  - Headset or microphone may be faulty.
- Batteries exhausted
  - fit new batteries.
- Incorrect operating mode.

### Low or distorted sound

- Batteries nearly exhausted
  - fit new batteries.

### Crackling or sound goes on or off

- Dirty plugs
  - clean plugs.
- Faulty leads
  - change leads.

### Tinny or muffled sound

- Tone control in wrong position.

The *Crescendo 50* is **NOT** designed to operate with Headphones or neckloops with mono jack plugs. Using these may cause damage to the amplifier.

## Maintenance and care

The *Crescendo 50* does not require any maintenance. If the unit does become dirty, simply wipe it clean with a soft, damp cloth. Never use spirits, thinners or other organic solvents. Do not set up the unit where it will be exposed to full sunlight for long periods. In addition it must be protected against excessive heat, moisture and severe mechanical shock.

## Warranty

The *Crescendo 50* is a very reliable product. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer.

The warranty covers the repair of the product and returning it to you free of charge during the warranty period. It is essential that you send in the product in its original packaging, so do not throw the packaging away. The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorized by Repairs under warranty are only carried out providing you have filled in and returned the enclosed warranty card from the dealer as well as a copy of the sales slip. Always specify the product number in any event.

## Disposal

Disposal of used electric and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system).

The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electric and electronic units. You protect the environment and health of your fellow men by the correct disposal of this products. Environment and health are endangered by a faulty disposal. Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of this product from your local community, your communal disposal company or your local dealer.

## Technical data

Dimensions:	26 x 62 x 95 mm (H x W x D)
Colour:	silver / grey
Weight:	120 g incl. batteries
Amplification:	max. 50 dB
Power supply:	2 x 1,5 V mignon batteries (type LR6 or AA)

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## **Humantechnik GHL AG**