HUMANTECHNIK

***DS-1***

**D**

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Seite 2

Digitalwecker DS-1

**GB**

**Operating Instructions**

Page 8

Alarm clock DS-1

**F**

**Mode d'emploi**

Page 14

Réveil digital DS-1

**NL**

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Pagina 20

Digitale wekker DS-1

**I**

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Pagina 26



Sveglia digitale DS-1

**E**

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Página 32

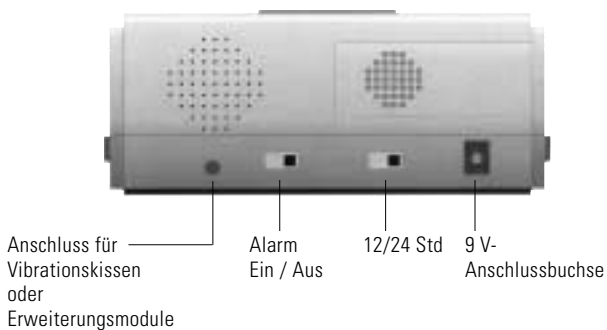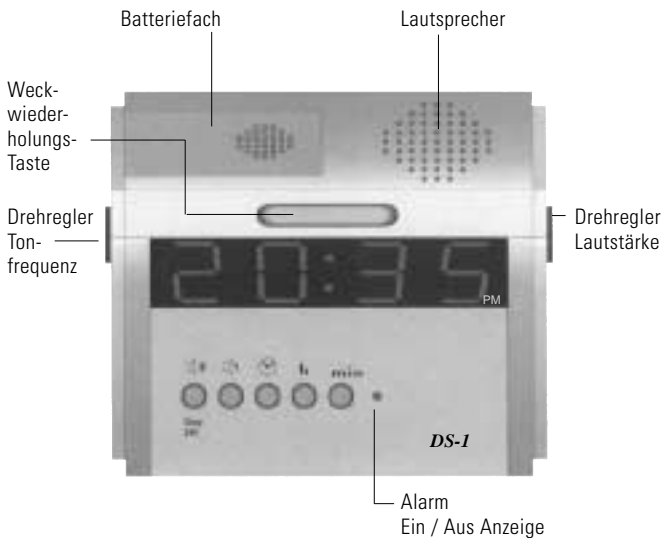Despertador digital DS-1

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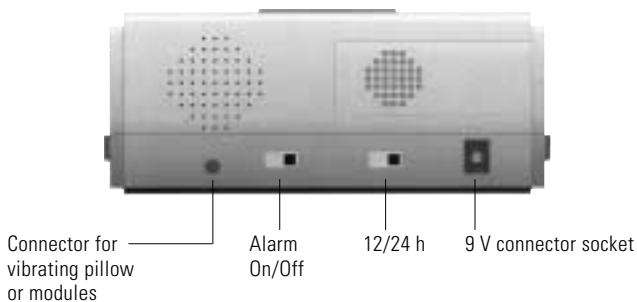
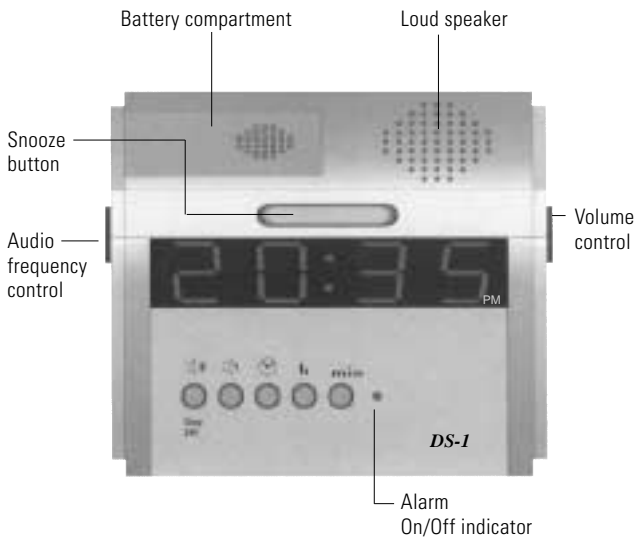
Congratulations on purchasing your alarm clock DS-1. You have selected a modern and reliable system. The alarm clock DS-1 is an electronic clock with a digital time display. It wakes you up with an acoustic signal which is adjustable in audio frequency and volume.

The following accessories can be connected to the alarm clock:

- Vibrating pillow
- Flash module
- Switch module
- Acoustic module

### Standard components

Please check if all following components are included:

- DS-1
- 9 V power supply unit
- Warranty card

### Installation

Connect the power supply unit to the 9 V connector socket of the alarm clock and plug it into a 230 V power outlet. The display flashes as a sign that the time has not yet been set. The flashing disappears as soon as the time is set.

### Time setting

Press  and **h** or **min** simultaneously to set the time.

#### Note:

The time can be displayed in a 12 or 24 hour modus. The setting can be changed at the 12/24 h switch at the back of the alarm clock.

### Setting the alarm time

The alarm clock has two alarm times that can be set separately.

#### Alarm time 1

Press  **1** and **h** or **min** simultaneously to set the desired alarm time (Duration of alarm signal 3 min).

#### Alarm time 2

Press  **2/ Stop 24 h** and **h** or **min** simultaneously to set the desired alarm time. (Duration of alarm signal 3 min).

#### Note

After the installation only alarm time 1 is active.

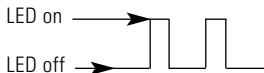
Both alarm times can be activated separately. Press **1** and **2/Stop 24 h** simultaneously and push the snooze button. With every push of the snooze button one of the three possible settings is shown on the display.

You can choose between the following settings:


Alarm time 1 active:  
Display indication: AL 1  
Alarm time 2 active:  
Display indication: AL - 2  
Alarm time 1 and 2 active:  
Display indication:  
AL 1 2

In addition to the display indication the LED flashes as follows:

#### Alarm time 1 aktiv:

#### Weckalarm 2 aktiv:

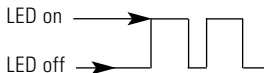Alarm time 1 and alarm time 2 active:

LED flashes continuously

### **Switching the alarm on/off**

Activate/deactivate the alarm with the alarm on/off switch at the back of the DS-1.

If the switch is in the OFF-position, both alarm times are deactivated.

### **Setting of the audio frequency**

The audio frequency of the alarm signal can be adjusted with the audio frequency control on the left side of the alarm clock. The audio frequency increases if the control is turned to the left and decreases if the control is turned to the right.

#### Note:

The alarm tone is turned off if the audio frequency control is turned completely to the left.

### **Setting of the volume**

The volume of the alarm signal can be changed with the volume control on the right side of the alarm clock. The volume decreases if the control is turned to the left, the volume increases if the control is turned to the right.

## Snooze function

Pressing the snooze button interrupts the alarm. The alarm starts again after approx. 5 minutes.

## Special features

- The alarm signal is repeated after 4 minutes (auto snooze) unless you press the snooze button or the Stop24h button.
- If a 9 V battery is inserted into the battery compartment the alarm clock operates even in case of a power failure. The display of the alarm clock turns off but the alarm signals are triggered.
- The inserted 9 V battery can be checked by pressing the snooze button. If "LO" is indicated on the display of the alarm clock, the battery has to be changed as otherwise the power failure back-up cannot be guaranteed.

## Maintenance and care

The DS-1 does not require any maintenance. If the unit does become dirty, simply wipe it clean with a soft, damp cloth. Never use spirits, thinners or other organic solvents. Do not set up the unit where it will be exposed to full sunlight for long periods. In addition it must be protected against excessive heat, moisture and severe mechanical shocks.

**Note:** This product is **not** protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, on or near the product.

## Disposal

Disposal of used electric and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system). The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electric and electronic units. You protect the environment and health of your fellow men by the correct disposal of this products. Environment and health are endangered by a faulty disposal. Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of this product from your local community, your communal disposal company or your local dealer.

## Warranty

The DS-1 is a very reliable product. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer.

This warranty covers the repair of the product and returning it to you free of charge. It is essential that you send in the product in its original packaging, so do not throw the packaging away.

The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people who are not authorised to do so (destruction of the seal on the unit).

Repairs under warranty are only carried out providing you have filled in and returned the enclosed warranty card from the dealer.

Always specify the product number in any event.

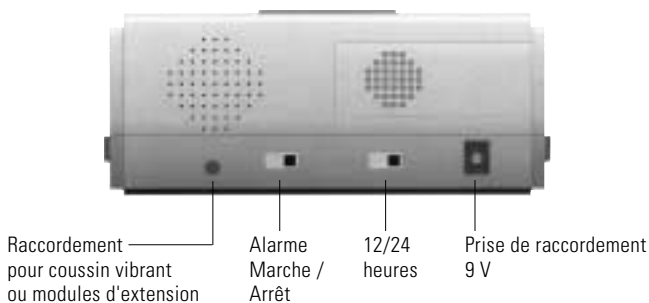
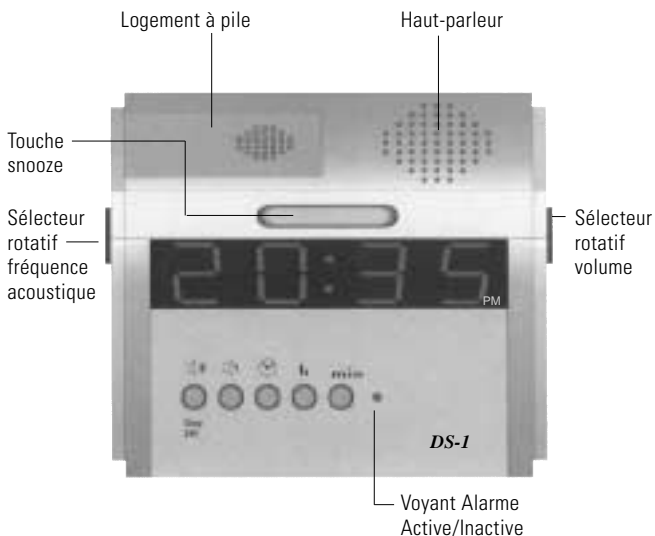
## Technical data

Power supply:

9 V power supply unit at 230 V, 50 Hz  
or 9 V battery

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## **Humantechnik GHL AG**
### **ELECTRONIQUE S.A.R.L**