HUMANTECHNIK

*InfraLight* DIR

**D**

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Seite 2

**GB**

**Operating Instructions**

Page 12

**F**

**Mode d'emploi**

Page 22

**NL**

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Pagina 32

**I**

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Pagina 42

**E**

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Página 52

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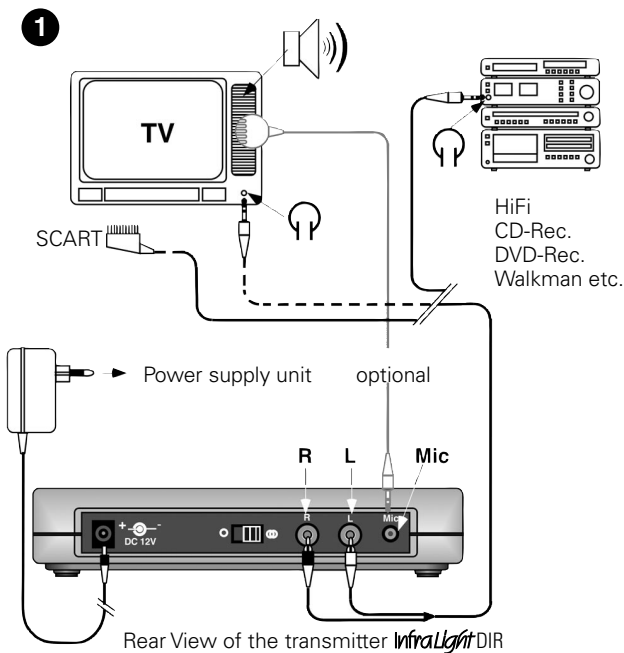
# Assembling the *InfraLight*<sup>DIR</sup> transmitter


The volume adjustment of the TV, HiFi, CD should be set to a comfortable listening level for other members of the household.

2


When you first charge the battery packs for the headset receiver let them charge for at least 14 hours. Only use *InfraLight* DIR battery packs.

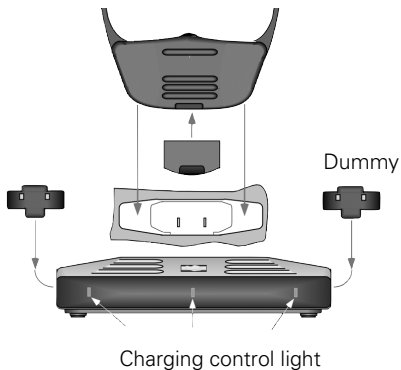The *InfraLight* DIR package includes two battery packs and one dummy.


3

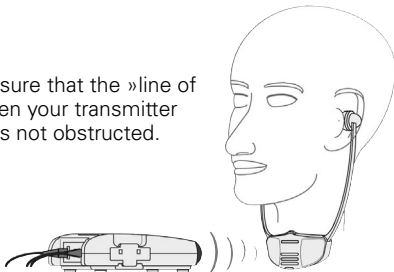Please make sure that the »line of sight« between your transmitter and receiver is not obstructed.


4
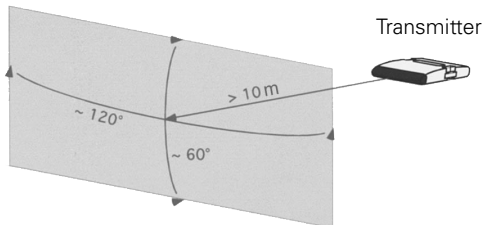
### Operating Range

The diagram below indicates the operating range of the *InfraLight* DIR. Reception will be cut off if the receiver is outside this area, approx. 10 m max. from transmitter.

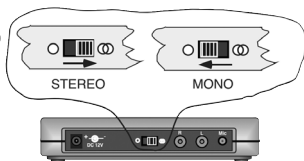
5

### Mono / Stereo Switch

**Note!** If the *InfraLight* DIR is set to »Stereo« but connected to an audio source via a mono plug, sound will only be heard in one ear piece. If this occurs move the switch to the »Mono« position.

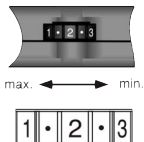6


### Volume adjustment of the *InfraLight* DIR receiver.

**There is no ON/OFF switch.**

The receiver automatically switches on when worn.

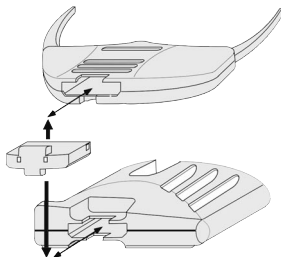7

If the sound quality of the receiver becomes noisy and distorted you should replace the battery pack with a charged one or charge the receiver, including battery, in its cradle.

**Warning: do not force the battery pack into the charger or receiver. It fits easily when in the correct orientation.**

8

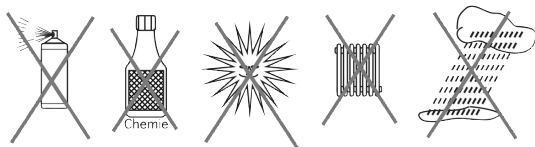
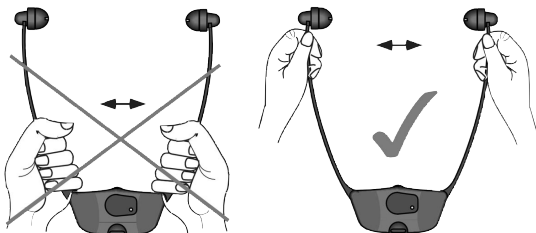
### **Maintenance of the headset receiver**

The ear pieces have to be cleaned or exchanged if dirty.

Please use for this the hygienic-set (order no. A-4982-0) which is available as a special accessory.

9

Please note the following information about your *InfraLight* DIR:

If the system needs cleaning **ONLY** use a soft damp cloth. If the foam ear pieces become damaged they should be replaced.

The product warranty does not cover normal wear or damage resulting from misuse, neglect, improper storage, accident, unauthorised repair or alteration.

### Important Notice

This product is **not** protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, on or near the product.

**10****Technical data:**

Modulation frequency	FM, Stereo and Mono
Carrier frequency	2,3 / 2,8 MHz
Frequency response	15 - 20000 Hz
Distortion	< 1 %
Noise level	typ. 60 dB
Battery charging time	approx. 14 h

**Headset receiver:**

Operating time per charge	approx. 6 h
Maximum volume SPL	approx. 120 dBA
Weight	47 g

**Transmitter:**

Power consumption approx.	3.6 VA
Operating voltage	12 Volt DC
Supply voltage	230 Volt 50 Hz
Weight	approx. 200 g

Technical specifications subject to change without notice.

Symptom	Cause	Solution
Recharging control lights. Don't light up when batteries are placed into the battery compartments properly.	Transmitter not connected to the power adapter or no power supply.	Make sure that the power adaptor is connected to the mains power socket and to the transmitter.
Recharging control light does not light up on the same side of the transmitter where a battery is inserted.	No contact has been made between the battery and the recharging contacts of the compartment of the transmitter.	The battery has not been correctly and completely inserted into the battery compartment.
The middle recharger control light does not light up despite having the receiver placed into its compartment in the transmitter.	No contact has been made between the battery in the receiver and the recharging contacts of the compartment in the transmitter.	The receiver has not been properly placed into the compartment. Make sure it is facing in the correct direction.
	There is no battery inserted into the receiver.	Insert a battery into the receiver.
Receiver doesn't work. No audio signals being received.	The transmitter is not correctly connected to the TV or HI-FI audio output.	Please check and make sure the audio connection cables to the TV or HI-FI are properly and correctly connected.
	Your TV or HI-FI equipment is not turned on.	Please read through the owner's manual for your TV or HI-FI equipment to make sure if the output for headphone or line output is connected and it does work correctly.

<b>Symptom</b>	<b>Cause</b>	<b>Solution</b>
Receiver doesn't work. No audio signals being received.	No battery is inserted into the receiver.	Insert a battery into the receiver.
	Battery is discharged	Replace battery with a fully charged one.
The audio transmission is of poor quality, noisy, distorted, unclear or no audio signals being received.	The "line of sight" between your transmitter and receiver is obstructed or you are too far away with your receiver from the transmitter.	Make sure that the front side of the transmitter is not obstructed and that it is a free "line of sight" transmission possibility. Make sure you are not located too far away from your transmitter.
	Battery is discharged.	Replace battery with a fully charged one.
	The system as used with a Plasma TV.	Plasma TV's disturb the infrared transmission. Please use one of our FM TV listening system.
Battery pack no longer gives the expected operating time per charge.	Battery pack is not fully charged.	Recharge the battery for the correct time.
	Battery pack has reached its recharging limit.	Replace battery pack with a new one.

## **Disposal**

Disposal of used electric and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system).

The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electric and electronic units. You protect the environment and health of your fellow men by the correct disposal of this products. Environment and health are endangered by a faulty disposal. Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of this product from your local community, your communal disposal company or your local dealer.

## **Warranty**

This warranty covers the repair of the product and returning it to you free of charge. It is essential that you send in the product in its original packaging, so do not throw the packaging away.

The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people who are not authorised to do so (destruction of the seal on the unit).

Repairs under warranty are only carried out providing you have filled in and returned the enclosed warranty card from the dealer.

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## **Humantechnik GHL AG**