HUMANTECHNIK

**LA-60**

**D**

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Ringschleifenverstärker

Seite 2

**GB**

**Operating Instructions**

Loop Amplifier

Page 8

**F**

**Mode d'emploi**

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## Operating elements

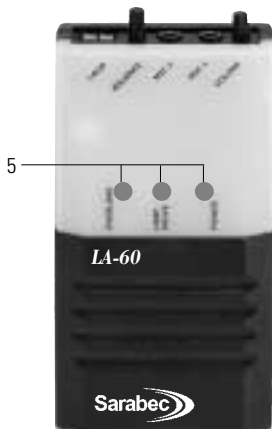
- 1 Volume control
- 2 Microphone connection
- 3 Balance control
- 4 Connection for loop pad or loop cable

A-4212-0 with loop pad

A-4213-0 with loop cable

- 5 Control LED's

Audio cables

Audio cables

We would like to congratulate you on purchasing your *LA-60 Loop Amplifier*. You have chosen a modern, reliable device.

Please first read these operating instructions carefully to initially start the device and to become familiar with all the system options.

## How it works

The LA-60 Loop Amplifier provides a practical solution for hearing aid users to listen more easily to conversation or other audio sources via the "T" or "MT" facility of their aid.

The amplifier unit picks up sound via a microphone or direct connection lead and sends the signal to a loop placed in the listening area which transmits the sound inductively, »magnetic sound waves«, to the hearing aid.

## Package contents

A-4212-0:

- *LA-60 Loop Amplifier*
- Loop pad
- Audio cable
- Operating instructions
- Guarantee card

A-4213-0:

- *LA-60 Loop Amplifier*
- Loop cable
- Audio cable
- Operating instructions
- Guarantee card

## Functions

- **POWER:** This light will be on constantly while power is being supplied to the system.
- **LOOP DRIVE:** This light shows that a signal is being passed from the amplifier unit to a loop and that the loop is continuous.
- **OVERLOAD:** If the red light is illuminated constantly, adjust the volume control unit the red light is just flickering giving the maximum undistorted output.
- **VOLUME:** This controls the volume level for the user.
- **BALANCE:**
  - This control balances the total level of the output signal between the two inputs.

- If using only one microphone ensure that you adjust this control to give the maximum output signal.
- Turn the balance control clockwise for channel 1 (mic 1), and anti-clockwise for channel 2 (mic 2).
- If using both microphones use the balance control to get optimum performance from both microphones.
- Each microphone channel can alternatively be used as a line input for connection to other audio equipment. (If using this method turn the balance control away from the input being used).

## **Operation**

- When using a microphone, either microphone socket can be used, adjust the balance control to gain the maximum output signal.
- To connect a loop pad push the exposed metal of the lead into the loop connectors ensuring that only the metal is gripped in the terminal. To release the cable press the grey buttons down with a screwdriver and pull the cable free.
- Before switching the unit on ensure that the volume control is set to its minimum level and the balance control is set to the middle position.
- Switch on the amplifier by pressing the POWER button on the mains socket, the green power light will then be illuminated.
- Ensure that the hearing aid wearer switches their aid to the the "T" or "MT" position.
- Adjust the volume control for best results.
- Adjust the balance control to optimise the signal from the microphone or sound source.

### Attention:

Before connection/disconnecting cables remove the LA-60 from the main socket.

## **What to do if problems occur**

### **No sound**

- Re-check all connections.
- Check green „POWER“ LED light is on.
- Microphone not plugged in or other audio source not connected.
- Hearing aid not on “T” / “MT” or “T” facility not working.

### **Low sound**

- Microphone too far from sound.
- Loop pad incorrectly placed. Check installation instructions.
- Volume control set too low. Turn up volume on amplifier.

### **Distored sound**

- Loop volume control too high. Turn volume down.
- Microphone plug/lead damaged.

### **Background noise (hum or buzz)**

- If noise remains when loop amplifier is turned off but hearing aid is still on “T”, this is interference caused by other equipment such as TV, fluorescent lights or dimmer switches. With the loop amplifier turned off and the hearing aid stiff on “T” turns items off in turn until the noise goes away.
- If noise stops when the loop amplifier is turned off there may be a fault in the system or microphone lead or noise is being picked up the microphone.

## **Maintenance and care**

The *LA-60* does not require any maintenance. If the unit does become dirty, simply wipe it clean with a soft, damp cloth. Never use spirits, thinners or other organic solvents. Do not set up the unit where it will be exposed to full sunlight for long periods. In addition it must be protected against excessive heat, moisture and severe mechanical shock.

## **Guarantee**

The warranty covers the repair of the product and returning it to you free of charge during the warranty period. It is essential that you send in the product in its original packaging, so do not throw the packaging away. The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorized by Repairs under warranty are only carried out providing you have filled in and returned the enclosed warranty card from the dealer as well as a copy of the sales slip. Always specify the product number in any event.

### **CE Marking**

This device complies with the requirements of EU directives: 73/23/EEC, 89/336/EEC and 1999/5/EC. Compliance with these directives is indicated by the CE Marking on the device.

## Disposal

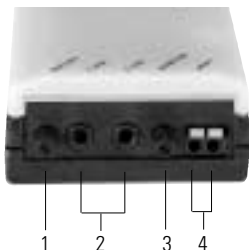
Disposal of used electric and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system).

The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electric and electronic units. You protect the environment and health of your fellow men by the correct disposal of this products. Environment and health are endangered by a faulty disposal. Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of this product from your local community, your communal disposal company or your local dealer.

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## **Humantechnik GHL AG**