HUMANTECHNIK

*lisa*

**D**

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Seite 2

Funk Alarmsender

**GB**

**Operating Instructions**

Page 8

Radio transmitter alarm

**F**

**Mode d'emploi**

Page 14

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**NL**

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Pagina 20

Draadloze Alarmzender

**I**

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Pagina 26

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

**E**

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Página 32

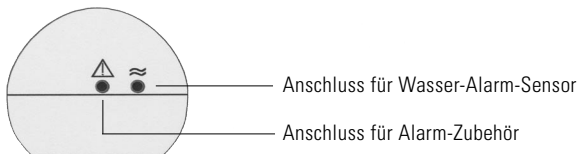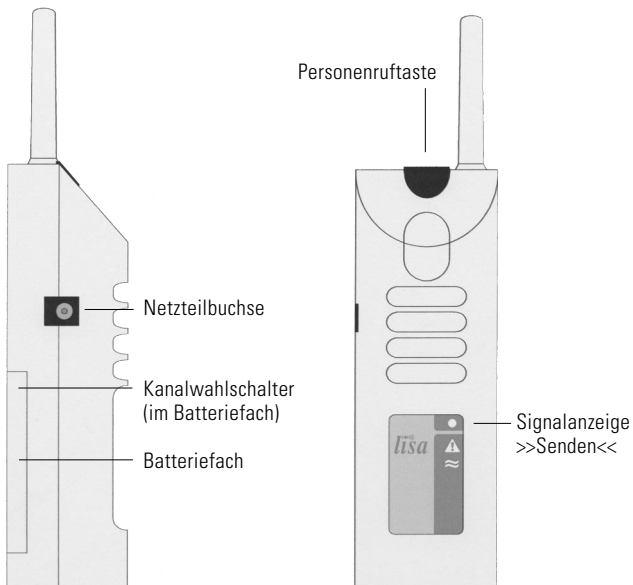Radioemisor de alarma

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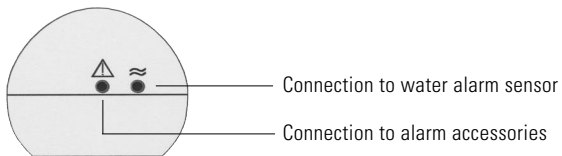
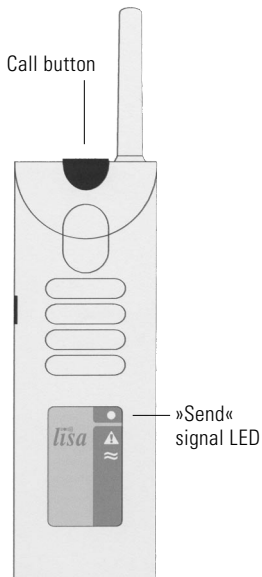
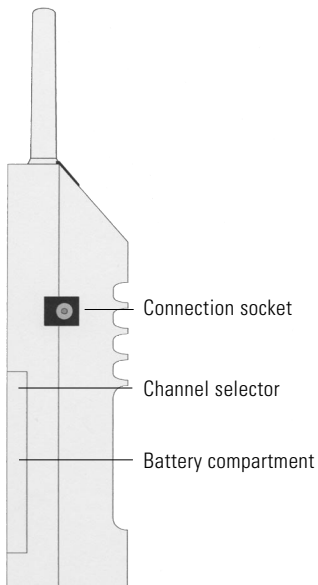
Congratulations on purchasing the *Radio Alarm Transmitter*. We hope it will give you many years of reassuring operation. Please read through these operating instructions carefully. They tell you how to set up the system correctly and explain all the system's features to you.

## Operating principle

A signalling system consists of at least one transmitter and one receiver. The *Radio Alarm transmitter* picks up the signals from various alarm sensors. It converts these signals into radio pulses (868.35 MHz) and transmits them to the receiver without the need for wires.

## Special features

When a signal is detected, it is transmitted for a total of 22 seconds. This increases both the reliability and the range of the system.

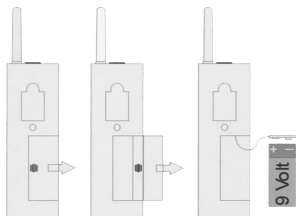
## Standard components

- *lisa Radio Alarm Transmitter*
- 9 V battery.
- Operating instructions
- Warranty card

If any parts are missing, please immediately contact your dealer or the manufacturer directly.

## Components not included

Alarm accessories

## Setting up the system –

### 1. Inserting the battery

There is a battery compartment on the back of the direct-connection *radio alarm transmitter*. Open it by sliding the cover of the battery compartment to the right of the transmitter using your thumb. Plug the terminals of a 9 V battery into the connection adapter. Insert the battery into the compartment and fit the cover back into place.

It is possible that the transmit signal may be triggered once when you connect the battery.

### 2. Setting the correct radio channel

A special digital transmission process ensures that the radio signals are picked up without errors. This process also makes it possible to set 10 different radio channels. When supplied from the factory, all *Radio Alarm Transmitters* are set to channel 0.

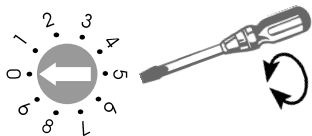
Fig. setting the correct radio channel

It is only necessary to set units to other channels if you have several different users operating units in the same building.

Set the channel you want by adjusting the channel selector switch on the left-hand side of the unit to the required channel number. Use a small screwdriver to adjust this switch.

All transmitters and receivers must be set to the same channel.

### 3. Installing the alarm accessories

Plug the alarm accessories into the socket on the underside of the *Radio Alarm Transmitter*.

This socket is identified by a triangle framing an exclamation mark:

#### Please note:

A separate socket must be used for the smoke detector.

This socket is marked with a flame.

#### **Available accessories:**

##### **Alarm mat (A-2921-0)**

Connect the plug from the alarm mat directly into the socket in the *Radio Alarm Transmitter* (alarm accessories socket). Place the alarm mat under the doormat.

##### **Alarm call button (A-2935-0)**

Connect the plug from the alarm call button directly into the socket in the *Radio Alarm Transmitter* (alarm accessories socket). The alarm signal is triggered by pressing the button (e.g. from the patient's bed).

##### **Water alarm sensor (A-2924-0)**

Plug the water alarm sensor into the socket of the transmitter marked with a double wave. The water alarm sensor is suitable for nearly all fluids. Stick the sensor with the supplied double-sided adhesive tape to the wall so that the contacts touch the floor. The signal is triggered as soon as both contacts are wetted by the fluid.

### **Mount-on contact (A-2922-0)**

For connecting to doors and windows. The mount-on contact consists of a permanent magnet and a reed contact. The contact is open when the magnet is in contact. An alarm signal is signalled if the magnet is separated from the contact. Use the signal cable to connect the mount-on contact to the *Radio Alarm Transmitter* (alarm accessories socket).

### **Smoke detector (A-2926-0)**

The smoke detector is triggered by smoke from smouldering fires and signals an alarm. Attach it to the ceiling of the room to be monitored and use the signal cable to connect the smoke detector to the *Radio Alarm Transmitter* (alarm accessories socket).

### **Call button**

Pressing the call button on the *Radio Alarm Transmitter* causes a signal to be triggered on every receiver which is set to the same radio channel as the transmitter. The call button lamp lights up to indicate this. This makes it possible to call people who are in different rooms. Also, this button can be used for checking the system is functioning even when the telephone cable is not connected.

### **Mains operation**

You can also operate your *Radio Alarm Transmitter* via the plug-in power supply unit. To connect the PSU, insert the connector on the end of the PSU cable into the corresponding socket on the left-hand side of the device. Then plug the power supply unit into the normal 230 V socket in your house.

### **Radio range**

Note the following points when using the *Radio Alarm Transmitter*: The radio signals are designed to have a range of 80 meters under optimum conditions.

The following conditions may result in a reduced range:

- Signal transmission through building structures or vegetation
- Interference from TVs, computer monitors and mobile phones.  
Under certain circumstances, this may prevent the receiver from functioning at all.
- Positioning of the transmitter at an unfavourable distance from surfaces which reflect radio waves, for example floors and walls.  
This may lead to the signal wave being weakened or even cancelled out by the reflected wave.
- Metallic objects reduce the range due to the shielding effect of the metal.
- Particularly in towns and cities, there may be many other sources

of radio waves which can disrupt the original signal.

- Having units operating at similar frequencies located close to one another may also result in mutual interference.

### **Positioning the radio transmitter**

Bear in mind the following points when positioning the radio transmitter, in order to achieve the greatest possible range from the radio transmitter:

- The radio transmitter should not be set up close to devices which emit powerful interference signals, such as TVs, computer monitors or mobile phones.
- The radio transmitter should not be placed on metallic objects, since otherwise the radio range will be restricted by metallic shielding.
- Position the radio transmitter as close to the centre of the building as possible, at a medium height in the room.

### **Maintenance and care**

The *Radio Alarm Transmitter* does not require any maintenance. If the unit does become dirty, simply wipe it clean with a soft, damp cloth. Never use spirits, thinners or other organic solvents.

Do not set up the *Radio Alarm Transmitter* where it will be exposed to full sunlight for long periods. In addition, it must be protected against excessive heat, moisture and severe mechanical shocks.

**Note:** This product is **not** protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, on or near the product.

### **Warranty**

The *Radio Alarm Transmitter* is a very reliable product. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer or the manufacturer directly.

This warranty covers the repair of the product and returning it to you free of charge. It is essential that you send in the product in its original packaging, so do not throw the packaging away.

The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorised to do so (destruction of the seal on the unit). Repairs will only be carried out under warranty if the completed warranty card is returned accompanied by a copy of the dealer's invoice/till receipt.

**Always specify the product number in any event.**

**Battery life** Under normal conditions (7 seven transmissions a day), the transmitter battery should last for about 1 year. Press the pager button on the transmitter to test the battery charge status. If the "Send" signal LED on the transmitter does not come on, the battery needs to be replaced!

## Disposal

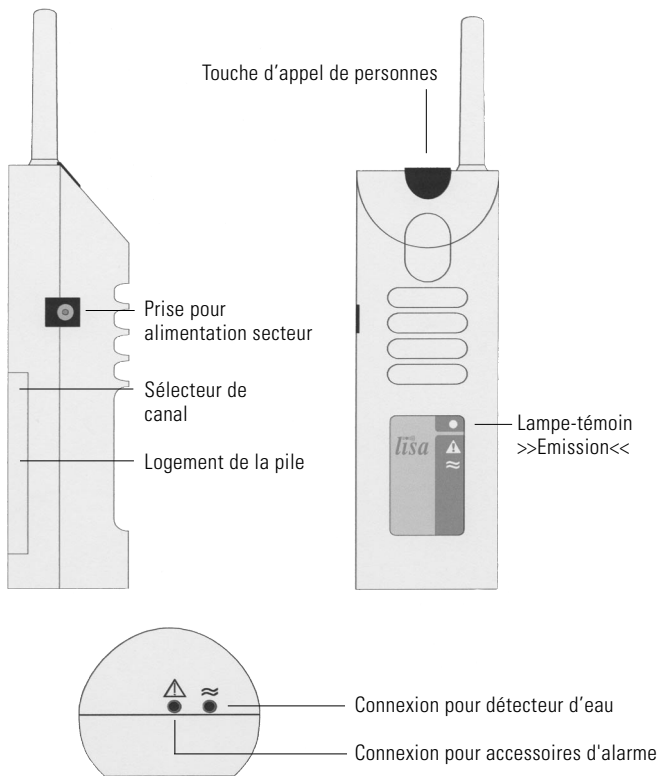
Disposal of used electric and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system). The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electric and electronic units. You protect the environment and health of your fellow men by the correct disposal of this products. Environment and health are endangered by a faulty disposal. Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of this product from your local community, your communal disposal company or your local dealer.

## Technical data

|                             |  |
|-----------------------------|--|
| Power supply:               | 9 V block battery or via a plug-in PSU |
| Battery life:               | approx. 1 year                         |
| Frequency:                  | 868.35 MHz                             |
| Operating range:            | 0° C – 40° C                           |
| Height                      |  |
| (including the antenna):    | 147 mm (201 mm)                        |
| Width:                      | 50 mm                                  |
| Depth:                      | 41 mm                                  |
| Weight (including battery): | 145 g                                  |

Technical specifications subject to change without notice.

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## **Humantechnik GHL AG**