HUMANTECHNIK

lisa

D

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Seite 2

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GB

Operating Instructions

Page 8

radio flashing receiver

F

Mode d'emploi

Page 14

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NL

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Pagina 20

Radio-flitsontvanger

I

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Pagina 26

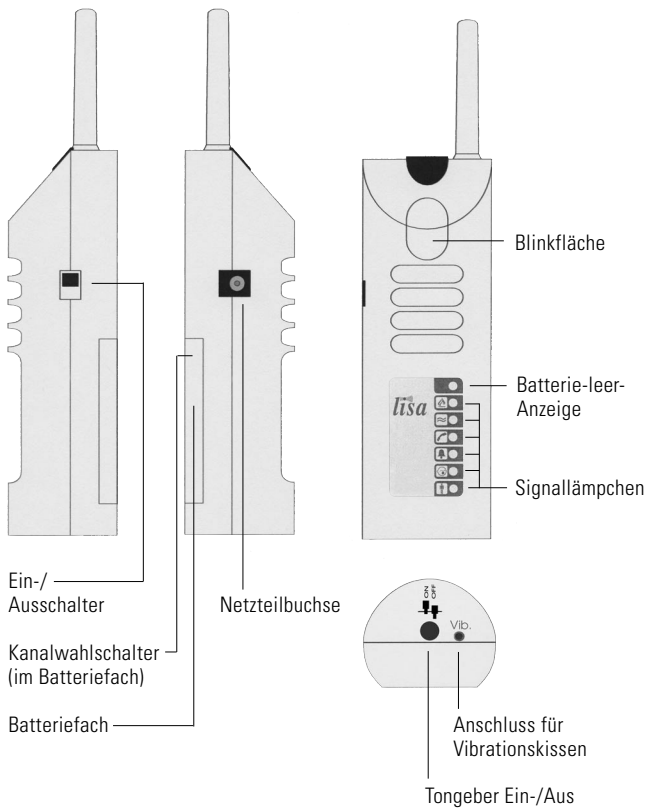
Ricevitore radio luminoso

E

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Página 32

Radorreceptor de luz intermitente

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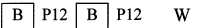
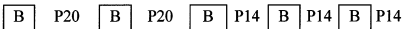
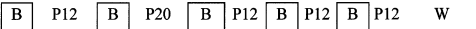
Blinkrhythmus  A sequence of four boxes: the first contains 'B', the second 'P12', the third 'B', and the fourth 'P12'. This is followed by a long horizontal line ending in 'W'.

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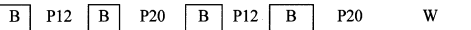
Blinkrhythmus  A sequence of six boxes: 'B P20', 'B P20', 'B P14', 'B P14', 'B P14'. This is followed by a long horizontal line.

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Blinkrhythmus  A sequence of five boxes: 'B P12', 'B P20', 'B P12', 'B P12', 'B P12'. This is followed by a long horizontal line ending in 'W'.

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Blinkrhythmus  A sequence of four boxes: 'B P12', 'B P20', 'B P12', 'B P20'. This is followed by a long horizontal line ending in 'W'.

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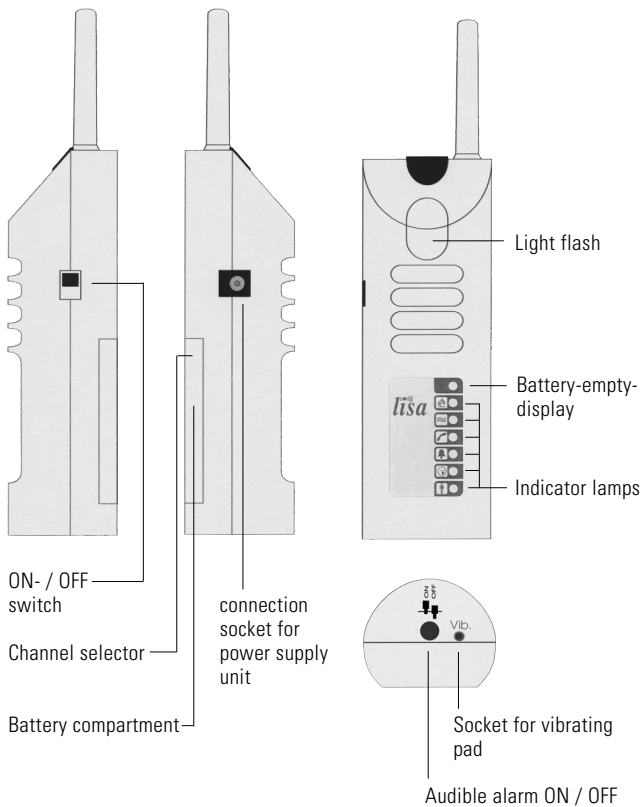
Congratulations on purchasing your *radio flashing receiver*. We hope it will provide you with many years of service. You have chosen a modern and reliable system.

Please read the operating instructions through carefully to be able to start the unit correctly and to familiarise yourself with all of the systems features.

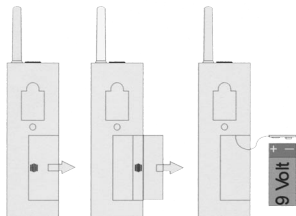
Functional principle

The *radio flashing receiver* receives signals from a variety of radio transmitters and converts them into light signals. The *radio flashing receiver* has a very high-power light-emitting diode (LED), which brings a signal to the user's attention, and additional LEDs which indicate the nature of the incoming signals.

Included in the standard package:

- *radio flashing receiver*
- 9 V block battery
- 9 V plug-in PSU
- Stand
- Operating Instructions
- Warranty card

If any parts are missing please immediately contact your dealer or the manufacturer directly.

Setting up the system –

1. Inserting the battery

There is a battery compartment on the back of the *radio flashing receiver*. Open it by sliding the cover of the battery compartment to the right of the receiver using your thumb. Connect a 9 V block battery to the connector plug, place the battery into the recess and close the receiver again. When the battery is connected, the receiving signals may be triggered once.

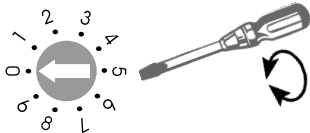
2. Setting the correct radio channel

A special digital transmission process ensures that the radio signals are picked up without errors. This process also makes it possible to set 10 different radio channels.

When supplied from the factory, all *radio flashing receivers* are set to channel 0. It is only necessary to set units to other channels if you have several different users operating units in the same building.

Set the required channel by using a small screwdriver to turn the channel selector switch, which is located towards the top of the battery compartment, to the required channel number.

All transmitters and receivers must be set to the same channel.

Operation –

1. Switching on the unit

Set the on/off switch to on. The *radio flashing receiver* is now ready to operate. After the unit has been switched on, each of the indicator lamps lights up briefly in succession and the receiver vibrates three times. This forms part of a self-test of the unit and lets you know that it is functioning correctly.

2. Transmission signal detection

The *radio flashing receiver* converts the various transmission signals into different sequences of vibration and lamp signals.

Telephone: Telephone LED lights

Flashing rhythm

B

 P14

B

 P30 W

Doorbell I: Doorbell LED lights

Flashing rhythm

B

 P14

B

 P14

B

 P14

B

 P40 W

Doorbell II: Doorbell LED lights

Flashing rhythm

B

 P20

B

 P20

B

 P20

B

 P40 W

Babymonitor: Babymonitor LED lights

Flashing rhythm

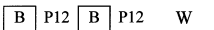
B

 P35

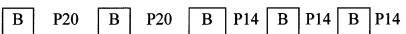
B

 P40 W

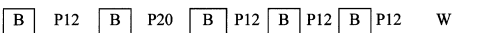
Pager: Pager LED lights

Flashing rhythm 

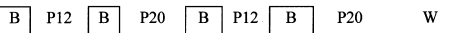
Alarm: All LEDs flash

Flashing rhythm 

Water alarm: Water alarm LED lights

Flashing rhythm 

Fire alarm: Fire alarm LED lights

Flashing rhythm 

P12 pause 1,2 seconds

P30 pause 3,0 seconds

P14 pause 1,4 seconds

P35 pause 3,5 seconds

P20 pause 2,0 seconds

P40 pause 4,0 seconds

B flash

The indicator LEDs remain on for 40 seconds.

Battery empty

If the “battery low” indicator lights up permanently, that means that the battery is almost dead. It’s time to replace or recharge it.

(The “battery low” indicator might flash briefly now and then. This doesn’t mean anything.)

Acoustic alarm

The acoustic alarm can be switched on or off using the on/off button.

Mains operation

You can also operate your *radio flashing receiver* via the plug-in power supply unit. To connect the PSU, insert the connector on the end of the PSU cable into the corresponding socket on the left-hand side of the device. Then plug the power supply unit into the normal 230 V socket in your house.

Radio range

Note the following points when using the *radio flashing receiver*:

The radio signals are designed to have a range of 80 meters under optimum conditions.

The following conditions may result in a reduced range:

- Signal transmission through building structures or vegetation
- Interference from TVs, computer monitors and mobile phones.
Under certain circumstances, this may prevent the receiver from functioning at all.
- Positioning of the transmitter at an unfavourable distance from surfaces which reflect radio waves, for example floors and walls.
This may lead to the signal wave being weakened or even cancelled out by the reflected wave.
- Metallic objects reduce the range due to the shielding effect of the metal.
- Particularly in towns and cities, there may be many other sources of radio waves which can disrupt the original signal.
- Having units operating at similar frequencies located close to one another may also result in mutual interference.

Maintenance and care

The *radio flashing receiver* does not require any maintenance. If the unit

does become dirty, simply wipe it clean with a soft, damp cloth. Never use spirits, thinners or other organic solvents.

Do not set up the *radio flashing receiver* where it will be exposed to full sunlight for long periods.

In addition, it must be protected against excessive heat, moisture and severe mechanical shocks.

Note: This product is **not** protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, on or near the product.

Warranty

The *radio flashing receiver* is a very reliable product. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer or the manufacturer directly.

This warranty covers the repair of the product and returning it to you free of charge. It is essential that you send in the product in its original packaging, so do not throw the packaging away.

The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorised to do so (destruction of the seal on the unit). Repairs will only be carried out under warranty if the completed

warranty card is returned accompanied by a copy of the dealer's invoice/till receipt.

Always specify the product number in any event.

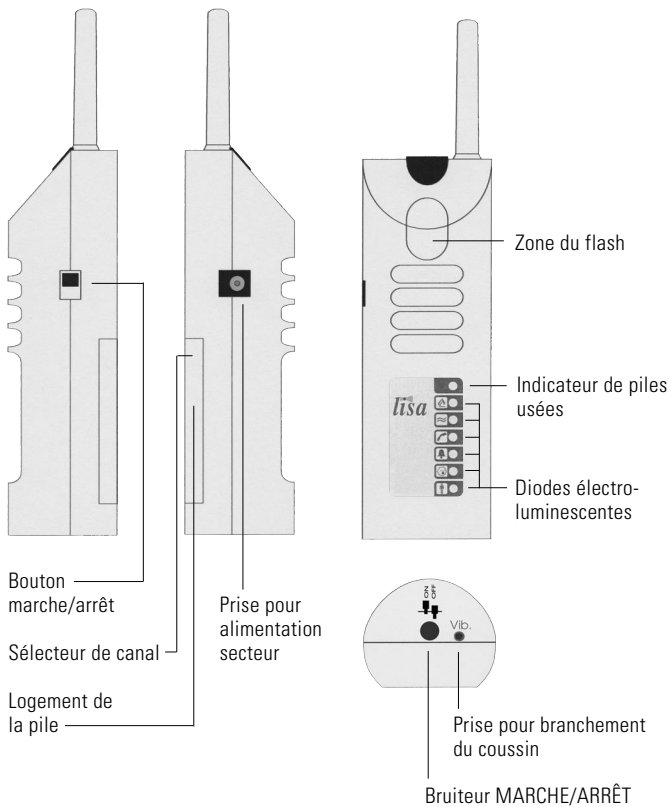
Disposal

Disposal of used electric and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system). The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electric and electronic units. You protect the environment and health of your fellow men by the correct disposal of this products. Environment and health are endangered by a faulty disposal. Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of this product from your local community, your communal disposal company or your local dealer.

Technical data

Power supply:	9 V block battery or via a plug-in PSU
Battery life:	Approx. 3-4 weeks (assuming 5 events per day)
Frequency:	868.35 MHz
Flash signaling:	1 LED high power with 2.5 W of power
Battery-low indicator:	1 LED orange
Visual signals:	2 LED red (fire/water alarm) 2 LED yellow (telephone/doorbell signal) 2 LED orange (baby phone/pager)
Height (including the antenna):	147 mm (201 mm)
Width:	50 mm
Depth:	41 mm
Weight (including battery):	145 g

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Humantechnik GHL AG
ELECTRONIQUE S.A.R.L