HUMANTECHNIK

lisa ((•))

D

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Seite 2

Blitzlampe

GB

Operating Instructions

Page 6

Flash lamp

F

Mode d'emploi

Page 10

Lampe-flash

NL

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Pagina 14

Flitslamp

I

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Pagina 18




Lampada flash

E

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Página 22

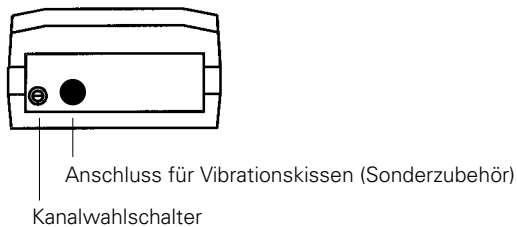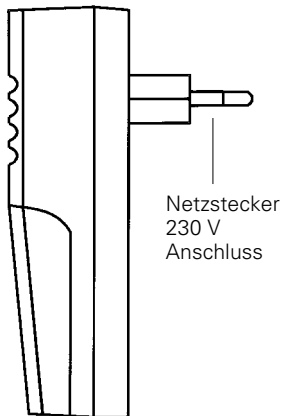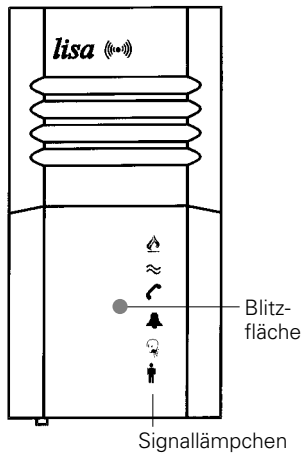Lámpara de destellos

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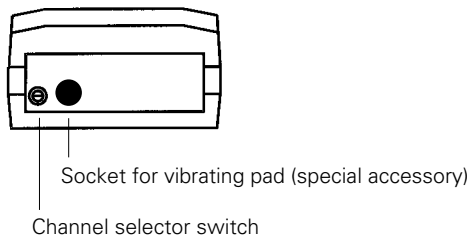
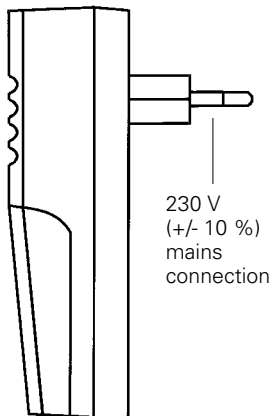
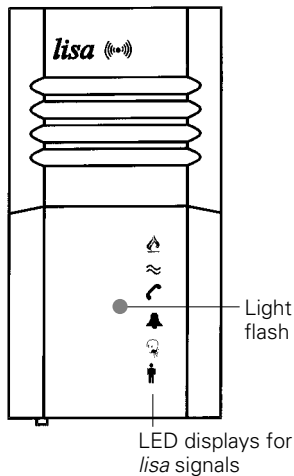
Congratulations on purchasing your cordless *flash lamp*. We hope it will provide you with many years of service. You have chosen a modern and reliable system.

Please read the operating instructions through carefully to be able to start the unit correctly and to familiarise yourself with all of the systems features.

Functional principle

The *flash lamp* receives signals from various transmitters and converts them into light signals. The *flash lamp* has a powerful flash tube which alerts you to a signal and LEDs displaying the type of signal being received.

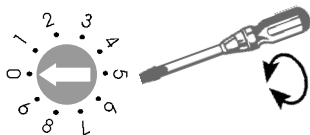
Included in the standard package:

- *flash lamp*
- Operating instructions
- document of warranty

If any parts are missing please immediately contact your dealer or the manufacturer directly.







Setting up the system – Setting the correct radio channel

A special digital transmission process ensures that the signals are picked up without any errors. This process also enables 10 different radio channels to be set. The default setting for the cordless *flash lamp* is channel 0. It is only necessary to set units to other channels if you have several different users operating units in the same building. Using a small screw driver, you can set the channel number you would like to use.

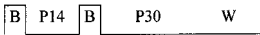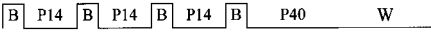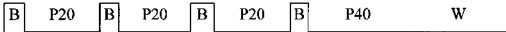Setting up the system – Installing the cordless *flash lamp*

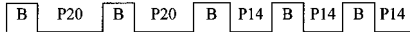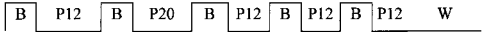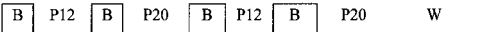The various transmission signals are converted into different sequences of flashes and signal displays by the cordless *flash lamp*.

Telephone:	Telephone LED lights
flashing sequence	
Doorbell I:	Doorbell LED lights
Blitzrhythmus	
Doorbell II:	Doorbell LED lights
flashing sequence	
Babymonitor:	Babymonitor LED lights
flashing sequence	
Pager:	Pager LED lights
flashing sequence	
Alarm:	All LEDs flash
flashing sequence	
Water alarm:	Water alarm LED lights
flashing sequence	
Fire alarm:	Fire alarm LED lights
flashing sequence	
P12	Pause 1,2 seconds
P20	Pause 2 seconds
P30	Pause 3 seconds
P40	Pause 4 seconds
B	flash

The indicator LEDs remain on for 40 seconds.

Maintenance and care

All *lisa* devices are maintenance-free. If the unit does become dirty, simply wipe it clean with a soft, damp cloth. Never use spirits, thinners or other organic solvents. Please do not expose *lisa* devices to direct sunlight for a long time and please protect them from excessive heat, moisture and severe mechanical shocks.

Note: This product is **not** protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, on or near the product.

Warranty

The *cordless flash lamp* is highly reliable. However, please contact your dealer or the manufacturer directly if a malfunction occurs in spite of the unit having been set up and operated correctly. This warranty covers the repair of the product and returning it to you free of charge. It is essential to return the phone in its original packaging, so do not throw this away. The warranty does not cover damage caused by incorrect handling or attempted repairs by unauthorised persons (destruction of the seal on the unit). Repairs will only be carried out under warranty if the dealer's warranty card has been filled in and returned or if you have the original warranty certificate if the phone was purchased directly from the manufacturer. The serial number must be specified in all cases.



Disposal

Disposal of used electric and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system). The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electric and electronic units. You protect the environment and health of your fellow men by the correct disposal of this products. Environment and health are endangered by a faulty disposal. Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of this product from your local community, your communal disposal company or your local dealer.

Technical data:

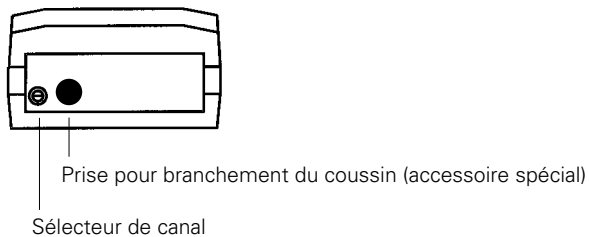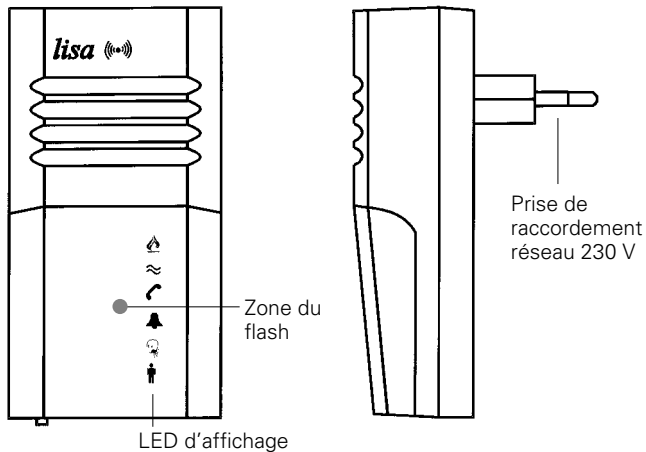Power supply:	230 V, 50 Hz,
Power consumpt. approx.: 1 W	
Frequency:	868,35 MHz
Height:	121 mm
Width:	65 mm
Depth:	35 mm
Weight:	215 g

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Humantechnik GHL AG
ELECTRONIQUE S.A.R.L