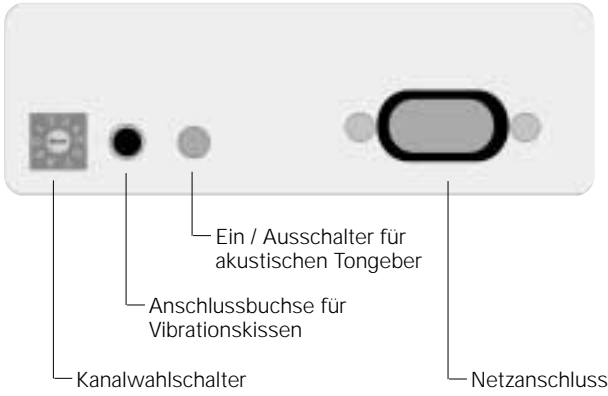
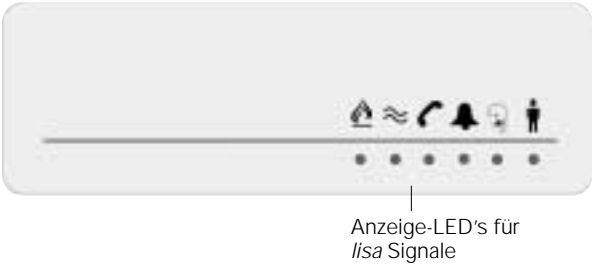
Tischblitzlampe
Table Flash Lamp
Lampe-flash de table
Tafelflitslamp
Lampada flash da tavolo
Lámpara de destellos
de mesa

Funk *lisa*
Radio

Bedienungsanleitung
Operating Instructions
Mode d'emploi
Gebruiksaanwijzing
Istruzioni per l'uso
Instrucciones de servicio

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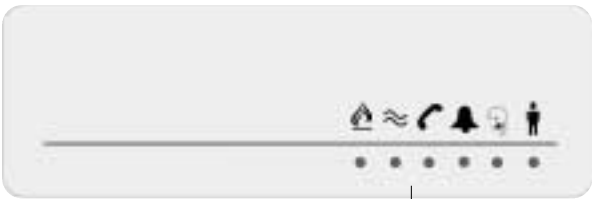
Indicator lamps for *lisa* signals

Channel selection switch
Connection socket for vibrating pad
On/off switch for acoustic signal
Mains connection

Congratulations on purchasing your *Radio table flash lamp*. We hope it will give you many years of reassuring operation. You have purchased a modern and reliable system. Please read through these operating instructions carefully. They tell you how to set up the system correctly and explain all the system's features to you.

Functional principle

The *Radio table flash lamp* receives the radio signals from various radio transmitters and converts them into light signals. It possesses a powerful flash tube which alerts you to a signal as well as LEDs which indicate the type of signals which are being received. In addition, it is possible to connect a vibrating pad (accessory).

Standard components

- *lisa Radio table flash lamp*
- 230 V mains lead

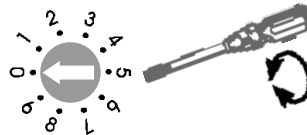
If any parts are missing, please immediately contact your dealer or the manufacturer directly.

Setting up the system – Setting the correct radio channel

A special digital transmission process ensures that the radio signals are picked up without errors. This process also makes it possible to set 10 different radio channels.

When supplied from the factory, all *Radio table flash lamps* are set to channel 0. It is only necessary to set units to other channels if you have several different users operating units in the same building.

Set the channel you want by adjusting the channel selector switch on the left-hand side of the unit to the required channel number. Use a small screwdriver to adjust this switch.

All transmitters and receivers must be set to the same channel.

Startup

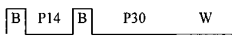
Installing the *Radio table flash lamp*


Simply connect the *lisa Radio table flash lamp* to a power point. The unit is immediately ready to use.


Make sure the flashing light can illuminate the room with as few obstacles as possible.


Transmission Signal Recognition:


The *Radio table flash lamp* converts the various transmission signals into varying sequences of flashes and signal displays.

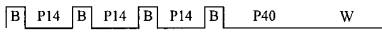
Telephone: The telephone LED lights up
Flashing sequence 

Doorbell I: The doorbell LED lights up
Flashing sequence 

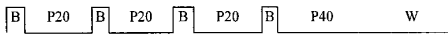Doorbell II: The doorbell LED lights up
Flashing sequence 

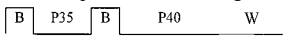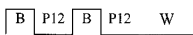
Baby monitor: The baby monitor LED lights up
Flashing sequence 

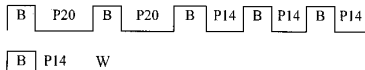Pager: The pager LED lights up
Flashing sequence 

Alarm: The telephone, door, baby alarm and pager LEDs flash
Flashing sequence 

Water alarm: The water alarm LED lights up

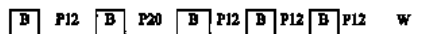Flashing
sequence

Fire alarm: Fire alarm LED

Flashing
sequence

P12	Pause 1.2 s
P14	Pause 1.4 s
P20	Pause 2.0 s
P30	Pause 3.0 s
P35	Pause 3.5 s
P40	Pause 4.0 s
B	Light flash

The indicator LEDs remain on for 40 seconds.

Power failure backup

The built-in battery can provide backup power for a mains power outage of up to 7 hours. The following functions are available during this period:

1. Acoustic signal
2. Triggering of indicator LEDs
3. Triggering of a vibration cushion (if connected)

Important: The flash light is not triggered during a power outage.

Maintenance and care

The *Radio table flash lamp* does not require any maintenance. If the unit does become dirty, simply wipe it clean with a soft, damp cloth. Never use spirits, thinners or other organic solvents.

Do not set up the *Radio table flash lamp* where it will be exposed to full sunlight for long periods. In addition, it must be protected against excessive heat, moisture and severe mechanical shocks.

Warranty

The *Radio table flash lamp* is a very reliable product. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer or the manufacturer directly.

This warranty covers the repair of the product and returning it to you free of charge. It is essential that you send in the product in its original packaging, so do not throw the packaging away.

The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authori-


sed to do so (destruction of the seal on the unit).

Repairs under warranty are only carried out providing you have filled in and returned the enclosed warranty card from the dealer or the original warranty certificate if you purchased the unit directly from the manufacturer. **Always specify the product number in any event.**

Technical Data

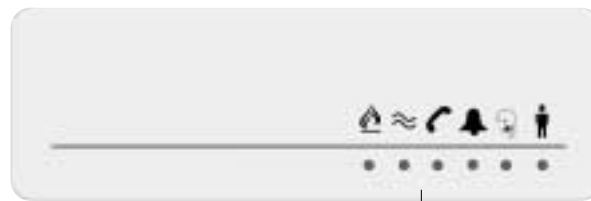Power supply:	230 V, 50 Hz,
Power consumption:	approx. 1 W
Frequency:	868,35 MHz
Height:	40 mm
Width:	120 mm
Depth:	135 mm
Weight:	approx. 350 g

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Humantechnik GHL AG
ELECTRONIQUE S.A.R.L