HUMANTECHNIK

lisa ((•))
RF time

D

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Seite 2

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
Gewicht: ca. 185 g

Light/Snooze // *lisa*
signal cancel switch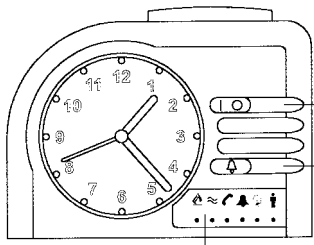


Alarm On/ Off
Push on/off
(green shows on)

Sound On/Off
Push On/Off
(green shows on)

Indicator LEDs
for *lisa* signals

Connection for
mains power
supply
unit

Socket for
vibrating pad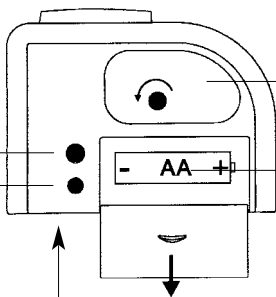

Time and Alarm
setting knob.
Pull to set time.

Warning:
NiCd battery only

channel selector
(underside)

Thank you for choosing the *lisa* RF time. You have purchased a modern and reliable alerting system. The *lisa* RF time alarm clock is an electronic clock with an analog dial. It is also a complete receiver for the *lisa* radio transmitters. It possesses a powerful vibrating pad, which alerts you to a signal, as well as 6 LEDs which indicate the type of signals that are being received.

Standard components

- *lisa* RF time
- vibrating pad
- 230 V power supply unit

Installing the *lisa* RF time

Open the battery compartment of the *lisa* RF time and remove the paper at the + contact of the battery. Connect the mains power supply unit to the *lisa* RF time and plug into a mains socket.

Time setting

Lightly pull the time and alarm setting knob and turn it in the direction of the arrow.

Warning: Pull the knob out carefully and only turn it in the direction of the arrow!

Alarm setting

Turn the time and alarm setting knob in the direction of the arrow.

Switching on the alarm

Push the Alarm On/Off button. When you see the green marking the alarm is on.

Switching on the sound

Push the Sound On/Off button. When you see the green marking the sound is on.

Snooze Alarm

Touching the Light/Snooze Alarm/*lisa* signal cancel switch will stop the alarm and it will start again in approximately 5 minutes.

Face illumination (back light)


Touching the Light/Snooze Alarm/*lisa* signal cancel switch will light the face for approx 4 seconds.

Setting the correct radio channel

A special digital transmission process ensures that the radio signals are received error-free. This process also makes it possible to set 10 different radio channels. When supplied from the factory, the *lisa* RF time is set to channel 0. It is only necessary to set units to other channels if you have several different users operating units in the same building. Set the channel you want by adjusting the channel selector switch on the underside of the unit

to the required channel number. Use a small screwdriver to adjust this switch.

All transmitters and receivers within your system must be set to the same channel.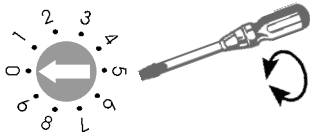

Transmission Signal Recognition

The *lisa* RF time converts the various transmission signals into varying sequences of vibrating and signal displays.

Telephone: Telephone LED lights

Vibrating-sequence _____

Doorbell I: Doorbell LED lights

Vibrating-sequence _____

Doorbell II: Doorbell LED flashes

Vibrating-sequence _____

Babymonitor: Baby Monitor LED lights

Vibrating-sequence _____

Pager: Pager LED lights

Vibrating-sequence _____


Alarm: All LEDs flash

Vibrating-sequence _____

Water alarm:

Water Alarm LED lights


Vibrating-
sequence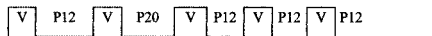

Fire alarm:

Fire Alarm LED lights

Vibrating-
sequence

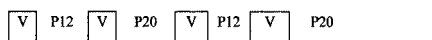P12 Pause 1,2 s

P14 Pasuse 1,4 s

P20 Pause 2 s

P30 Pause 3 s

P40 Pause 4 s

V Vibration

The indicator LEDs remain on for 40 seconds.

Touching the Light/Snooze switch cancel any vibration caused by *lisa* signals. The indicator LEDs light the normal time (40 seconds).

Recalling the most recent signal

The *lisa* RF time stores each signal it receives for a period of 3 minutes.

During this time, you can call up the signal again by pressing the Light/Snooze /*lisa* signal cancel switch button. The signal is then displayed on the indicator LEDs. A vibration signal is not triggered.

Radio range

The *lisa* transmitters are designed to have a radio signal range of 80 meters under optimum conditions.

The following conditions may result in a reduced range:

- Signal transmission through building structures or vegetation

- Interference from TVs, computer monitors and mobile phones. Under certain circumstances, this may prevent the receiver from functioning at all.
- Positioning of the transmitter at an unfavorable distance from surfaces which reflect radio waves, for example floors and walls.

This may lead to the signal wave being weakened or even cancelled out by the reflected wave.

- Metallic objects reduce the range due to the shielding effect of the metal.
- Particularly in towns and cities, there may be many other sources of radio waves which can disrupt the original signal.
- Having units operating at similar frequencies located close to one another may also result in mutual interference.

Power failure backup

The battery can provide backup power for a mains power outage of up to 72 hours. All functions are available during this period.

Flat battery indicator

If the pager LED lights up without a *lisa* signal the battery voltage is too low.

Symptom: Flat battery indicator (Pager LED) lights up permanently.

Cause: Power supply unit isn't connected to the *lisa* RF time battery is faulty - replace battery with a new one.

Attention: Use only a NiCd battery. After the unit has been disconnected from the mains supply for several months it must be plugged into the mains supply for several hours before the unit will function reliably on batteries again.

Maintenance and care

The *lisa* RF time does not require any maintenance. If the unit does become dirty, simply wipe it clean with a soft, damp cloth. Never use spirits, thinners or other organic solvents. Do not set up the *lisa* RF time where it will be exposed to full sunlight for long periods. In addition, it must be protected against excessive heat, moisture and severe mechanical shocks.

Note: This product is **not** protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, on or near the product.

Warranty

The *lisa* RF time is a very reliable product. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer or the manufacturer directly. This warranty covers the repair of the product and returning it to you free of charge during the warranty period. It is essential that you send in the product in its original packaging, so do not throw the packaging away. The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorized by Humantechnik to do so (do not break the seal on the unit).

Repairs under warranty are only carried out providing you have filled in and returned the enclosed warranty card from the dealer or the original warranty certificate from Human-technik if you purchased the unit directly from the manufacturer. Always specify the product number in all cases.

Disposal

Disposal of used electrical and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system).

The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electrical and electronic units.

You protect the environment and health of your fellow man by the correct disposal of this product. Environment and health are endangered by incorrect disposal. Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of products from your local community, your communal disposal company or your local dealer.

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Humantechnik GHL AG
ELECTRONIQUE S.A.R.L