HUMANTECHNIK

*lisa* ((•))  
RF time *flash*

**D**

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Seite 2

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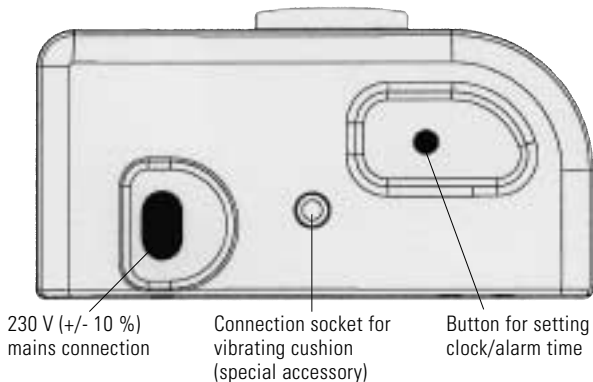
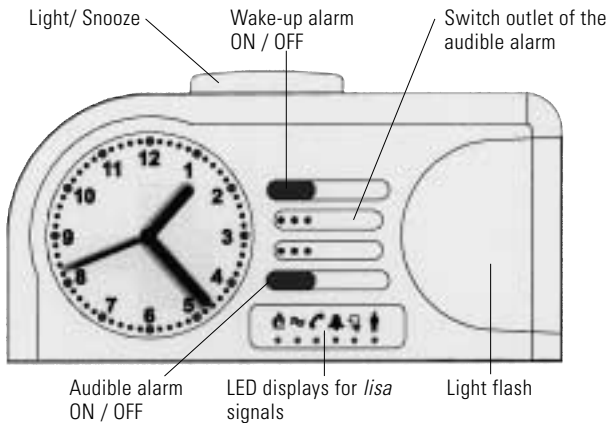
Congratulations on purchasing your *lisa* RF time flash. You have selected a modern and reliable system.

The *lisa* RF time flash is an electronic clock with an analogue time display and a receiver for the radio signals from various *lisa* radio transmitters. When a radio signal is detected, you are alerted in the form of a flashing light / acoustic alarm and 6 light-emitting diodes. It is also possible to connect a vibrating cushion (special accessory, order no. A-3303-0).

### **Standard components**

- *lisa* RF time flash
- 230 V power supply unit
- Warranty card

### **Startup**

Connect the supplied mains lead to the socket on the alarm clock and plug it into a 230 V (+/- 10 %) power outlet.

### **Time setting**

Lightly pull the time and alarm setting knob and turn it in the direction of the arrow.

Warning: Pull the knob out carefully and only turn it in the direction of the arrow!

### **Alarm setting**

Turn the time and alarm setting knob in the direction of the arrow.

### **Switching on the alarm**

Push the Alarm On/Off button. When you see the green marking the alarm is on.

### **Switching on the sound**

Push the Sound On/Off button. When you see the green marking the sound is on.

### **Snooze Alarm**

Touching the Light/Snooze Alarm/*lisa* signal cancel switch will stop the alarm and it will start again in approximately 5 minutes.

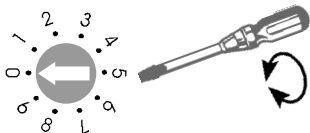
### **Face illumination (back light)**

Touching the Light/Snooze Alarm/*lisa* signal cancel switch will light the face.

### **Setting the correct radio channel**

A special digital transmission process ensures that the radio signals are received error-free. This process also makes it possible to set 10 different radio channels. When supplied from the factory, the *lisa* RF time flash is set to channel 0. It is only necessary to set units to other channels if you have several different users operating units in the same building. Set the channel you want by adjusting the channel selector switch on the underside of the unit to the required channel number.

Use a small screwdriver to adjust this switch.  
 All transmitters and receivers within your system must be set to the same channel.

### Transmission Signal Recognition

The *lisa* RF time flash converts the various transmission signals into varying sequences of vibrating and signal displays.

#### Telephone: Telephone LED lights

Vibrating-sequence V P14 V P30 \_\_\_\_\_

#### Doorbell I: Doorbell LED lights

Vibrating-sequence V P14 V P14 B P14 V P40 \_\_\_\_\_

#### Doorbell II: Doorbell LED flashes

Vibrating-sequence V P20 V P20 V P20 V P40 \_\_\_\_\_

#### Babymonitor: Baby Monitor LED lights

Vibrating-sequence V P35 V P40 \_\_\_\_\_

#### Pager: Pager LED lights

Vibrating-sequence V P12 V P12 \_\_\_\_\_

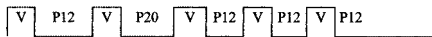
#### Alarm: All LEDs flash

Vibrating-sequence V P20 V P20 V P14 V P14 V P14  
V P14 \_\_\_\_\_

## Water alarm:

Water Alarm LED lights


Vibrating-  
sequence

## Fire alarm:

Fire Alarm LED lights

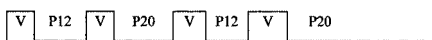Vibrating-  
sequence

P12	Pause 1,2 s	P30	Pause 3 s
P14	Pause 1,4 s	P40	Pause 4 s
P20	Pause 2 s	V	Vibration

The indicator LEDs remain on for 40 seconds.

Touching the Light/Snooze switch cancel any vibration caused by *lisa* signals. The indicator LEDs light the normal time (40 seconds).

## Recalling the most recent signal

The *lisa* RF time flash stores each signal it receives for a period of 3 minutes. During this time, you can call up the signal again by pressing the Light/Snooze /*lisa* signal cancel switch button. The signal is then displayed on the indicator LEDs. A vibration signal is not triggered.

## Radio range

The *lisa* transmitters are designed to have a radio signal range of 80 meters under optimum conditions.

The following conditions may result in a reduced range:

- Signal transmission through building structures or vegetation
- Interference from TVs, computer monitors and mobile phones. Under certain circumstances, this may prevent the receiver from functioning at all.
- Positioning of the transmitter at an unfavorable distance from surfaces which reflect radio waves, for example floors and walls. This may lead to the signal wave being weakened or even cancelled out by the reflected wave.

- Metallic objects reduce the range due to the shielding effect of the metal.
- Particularly in towns and cities, there may be many other sources of radio waves which can disrupt the original signal.
- Having units operating at similar frequencies located close to one another may also result in mutual interference.

### Power failure backup

The rechargeable battery in the unit can provide backup power for a period of up to 8 hours without mains power. Almost all of the functions of the *lisa* RF time flash are retained during this period.

### Flat battery indicator

If the voltage of the internal rechargeable battery is too low if the pager display lights up although there is no signal present. Connect the alarm clock to the mains power. Important: The *lisa* RF time flash must be operated from the mains power for a few hours at first after it has been in storage for several months, e.g. in the shop, before all of its functions become active (necessary for the internal battery to be recharged).

### Special aspects

- The alarm signal is repeated after 4 minutes (Autosnooze) unless you

press the Light/Snooze button or the Wake-up alarm ON / OFF button.

- The clock face illumination is also activated when the alarm is triggered.
- As well as the green marking of the Alarm ON / OFF switch, the Snooze button is also illuminated when the alarm is triggered.
- You can select between two different alarm types. The alarm type is altered as described below:
  - Switch off the alarm -- Press and hold the Snooze button for 3 seconds
  - Switch the alarm back on --
 The Snooze button flashes to indicate that the second alarm type has been selected.

### Maintenance and care

The *lisa* RF time flash does not require any maintenance. If the unit does become dirty, simply wipe it clean with a soft, damp cloth. Never use spirits, thinners or other organic solvents. Do not set up the *lisa* RF time flash where it will be exposed to full sunlight for long periods. In addition, it must be protected against excessive heat, moisture and severe mechanical shocks.

**Note:** This product is **not** protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, on or near the product.

## Warranty

The *lisa* RF time flash is a very reliable product. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer or the manufacturer directly. This warranty covers the repair of the product and returning it to you free of charge during the warranty period. It is essential that you send in the product in its original packaging, so do not throw the packaging away. The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorized to do so (do not break the seal on the unit).

Repairs under warranty are only carried out providing you have filled in and returned the enclosed warranty card from the dealer or the original warranty certificate if you purchased the unit directly from the manufacturer.

Always specify the product number in all cases.

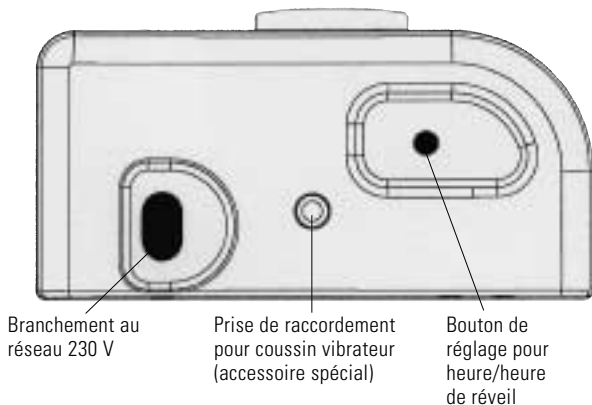
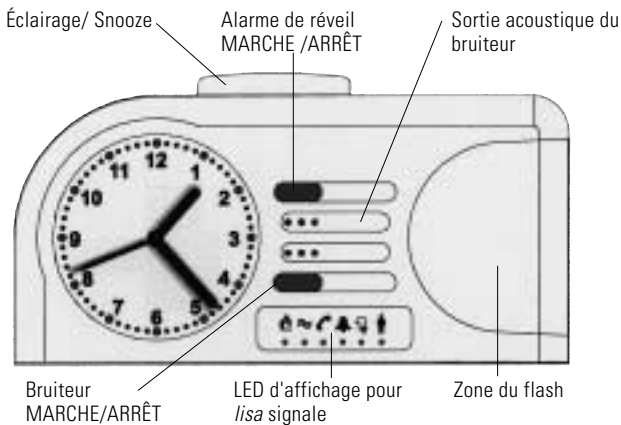
## Disposal

Disposal of used electrical and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system). The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electrical and electronic units. You protect the environment and health of your fellow man by the correct disposal of this product. Environment and health are endangered by incorrect disposal. Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of products from your local community, your communal disposal company or your local dealer.

## Technical data

Power supply:	230 V, 50 Hz
Frequency:	868.35 MHz
Power consumpt.	3 W

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**Humantechnik GHL AG**
**ELECTRONIQUE S.A.R.L**