HUMANTECHNIK

lisa

D

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Seite 2

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GB

Operating Instructions

Page 8

Portable Radio Receiver

F

Mode d'emploi

Page 14

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NL

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Pagina 20

Draagbare Radio-Ontvanger

I

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Pagina 26

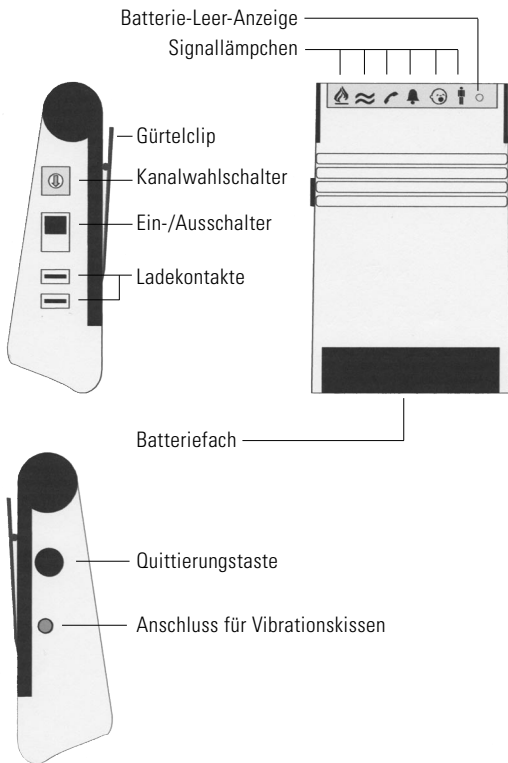
Ricevitore radio portatile

E

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Página 32

Radorreceptor portátil

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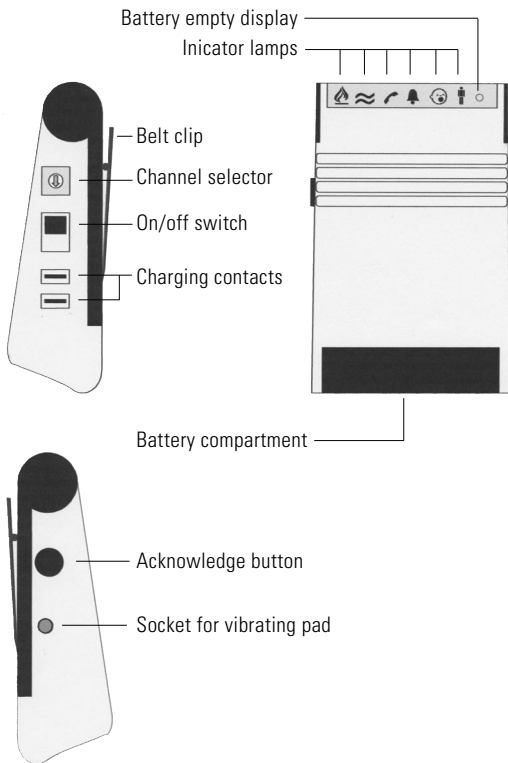
Congratulations on purchasing your *Portable Radio Receiver*. We hope it will give you many years of reassuring operation.

You have purchased a modern and reliable system.

Please read through these operating instructions carefully. They tell you how to set up the system correctly and explain all the system's features to you.

Operating principle

The *Portable Radio Receiver* receives the signals from various radio transmitters.

It converts these signals into vibrating pulses. In addition, 6 indicator lamps help you to different identify signals.

Standard components

- *lisa Portable Radio Receiver*
- Carry cord,
- 2 x 1.5 V AAA batteries
- Operating instructions
- Warranty card

If any parts are missing, please immediately contact your dealer or the manufacturer directly.

Controls

Indicator lamps: The 6 indicator lamps help you to differentiate between the individual transmitter signals. They light for a total of 40 seconds.

Flat battery indicator: The LED flashes to display that the battery is nearly flat. You need to fit a new battery.

Reset button: When you feel the *Portable Radio Receiver* vibrating, you can switch off its vibration function using the reset button on the receiver. The indicator lamps light and go out automatically after a total of 40 seconds.

Furthermore, you can press the acknowledge button to call up the most recent signal again (the function works for up to 3 minutes after receipt of the signal).

Belt clip: You can use the belt clip to attach the *Portable Radio Receiver* to your clothing.

On/off switch: This switch turns the *Portable Radio Receiver* on and off.

Channel selector: You can select the radio channel using the channel selector.

Setting up the system – 1. Inserting the battery

There is a battery compartment on the underside of the portable radio receiver. Open it by sliding the cover of the battery compartment away from the receiver using both your thumbs. Insert two 1.5 V AAA batteries and close the receiver again.

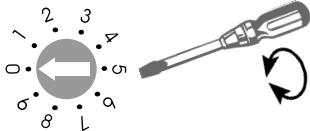
2. Setting the correct radio channel

A special digital transmission process ensures that the radio signals are picked up without errors. This process also makes it possible to set 10 different radio channels.

When supplied from the factory, all *Portable Radio Receivers* are set to channel 0. It is only necessary to set units to other channels if you have several different users operating units in the same building.

Set the required channel by using a small screwdriver to turn the channel selector switch, which is located on the right of the receiver, to the required channel number. Use a small screwdriver to adjust this switch.

All transmitters and receivers must be set to the same channel.

Operation –

1. Switching on the unit

Set the on/off switch to on. The *Portable Radio Receiver* is now ready to operate. After the unit has been switched on, each of the indicator lamps lights up briefly in succession and the receiver vibrates three times. This forms part of a self-test of the unit and lets you know that it is functioning correctly.

2. Transmission signal detection

The *Portable Radio Receiver* converts the various transmission signals into different sequences of vibration and lamp signals.

Telephone: The telephone light-emitting diode (LED) lights up for 40 seconds. The vibration (28 seconds) sequence is as follows:

Vibration

| | | | |
|----|----|----|-----|
| V4 | P8 | V4 | P30 |
|----|----|----|-----|

Door I: The door LED lights up for 40 seconds. The vibration (28 seconds) sequence is as follows:

Vibration

| | | | | | | | |
|----|----|----|----|----|----|----|-----|
| V4 | P8 | V4 | P8 | V4 | P8 | V4 | P36 |
|----|----|----|----|----|----|----|-----|

Door II: The door LED lights up for 40 seconds. The vibration (28 seconds) sequence is as follows:

Vibration

| | | | | | | | |
|----|-----|----|-----|----|-----|----|-----|
| V4 | P16 | V4 | P16 | V4 | P16 | V4 | P36 |
|----|-----|----|-----|----|-----|----|-----|

Baby monitor: The baby LED lights up for 40 seconds. The vibration (28 seconds) sequence is as follows:

Vibration

| | | | |
|----|-----|----|-----|
| V4 | P16 | V4 | P48 |
|----|-----|----|-----|

Call button: The call button LED lights up for 40 seconds.

The vibration (28 seconds) sequence is as follows:

Vibration

| | | | |
|----|----|----|----|
| V4 | P8 | V4 | P8 |
|----|----|----|----|

Alarm: All four LEDs flash for 40 seconds. The vibration (28 seconds) sequence is as follows:

Vibration

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Water alarm: Water alarm LED lights

Vibration

| | | | | | | | | | |
|----|-----|----|-----|----|-----|----|-----|----|-----|
| V4 | P12 | V4 | P20 | V4 | P12 | V4 | P12 | V4 | P12 |
|----|-----|----|-----|----|-----|----|-----|----|-----|

Fire alarm: Fire alarm LED lights

Vibration

| | | | | | | | |
|----|-----|----|-----|----|-----|----|-----|
| V4 | P12 | V4 | P20 | V4 | P12 | V4 | P20 |
|----|-----|----|-----|----|-----|----|-----|

V4 vibration 0,4 seconds

P8 pause 0,8 seconds

P12 pause 1,2 seconds

P16 pause 1,6 seconds

P20 pause 2,0 seconds

P30 pause 3 seconds

P36 pause 3,6 seconds

P48 pause 4,8 seconds

Battery empty

The battery empty LED lights up. The batteries need to be exchanged.

3. Recalling the most recent signal

The *Portable Radio Receiver* stores each signal it receives for a period of 3 minutes. During this time, you can call up the signal again by pressing the reset button. The signal is then displayed on the indicator lamps. A vibration signal is not triggered.

Radio range

Note the following points when using the *Portable Radio Receiver*:

The radio signals are designed to have a range of 80 meters under optimum conditions.

The following conditions may result in a reduced range:

- Signal transmission through building structures or vegetation
- Interference from TVs, computer monitors and mobile phones.
Under certain circumstances, this may prevent the receiver from functioning at all.
- Positioning of the transmitter at an unfavourable distance from surfaces which reflect radio waves, for example floors and walls.
This may lead to the signal wave being weakened or even cancelled out by the reflected wave.
- Metallic objects reduce the range due to the shielding effect of the metal.
- Particularly in towns and cities, there may be many other sources of radio waves which can disrupt the original signal.
- Having units operating at similar frequencies located close to one another may also result in mutual interference.

Maintenance and care

The *Portable Radio Receiver* does not require any maintenance. If the

unit does become dirty, simply wipe it clean with a soft, damp cloth.

Never use spirits, thinners or other organic solvents.

Do not set up the *Portable Radio Receiver* where it will be exposed to full sunlight for long periods.

In addition, it must be protected against excessive heat, moisture and severe mechanical shocks.

Note: This product is **not** protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, on or near the product.

Warranty

The *Portable Radio Receiver* is a very reliable product. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer or the manufacturer directly.

This warranty covers the repair of the product and returning it to you free of charge. It is essential that you send in the product in its original packaging, so do not throw the packaging away.

The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorised to do so (destruction of the seal on the unit). Repairs will only be carried out under warranty if the completed warranty card is returned accompa-

nied by a copy of the dealer's invoice/till receipt.
Always specify the product number in any event.

Disposal

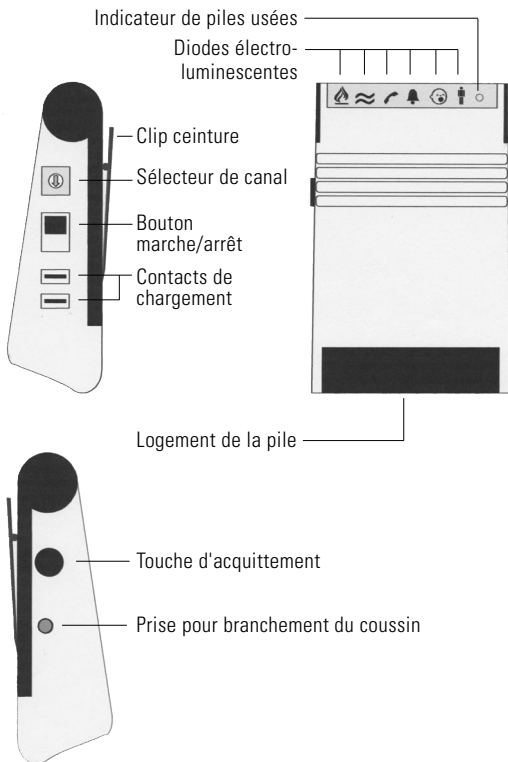
Disposal of used electric and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system). The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electric and electronic units. You protect the environment and health of your fellow men by the correct disposal of this products. Environment and health are endangered by a faulty disposal. Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of this product from your local community, your communal disposal company or your local dealer.

Technical data

| | |
|---------------------------|---|
| Power supply: | 1.5 V Mignon battery or 1.2 V rechargeable battery approx. 800 h with 1.5 V alka-line battery approx. 400 h with 1.5 V zinc carbon battery approx. 300 h with 1.2 V rechargeable battery |
| Number of radio channels: | 10 |
| Frequency: | 868.35 MHz |
| Optical signals: | 6 red LEDs |
| Flat battery indicator: | 1 green LED |
| Vibration: | Eccentric motor, different vibration sequences to differentiate between signals |
| Height: | 85 mm |
| Width: | 60 mm |
| Depth: | 28 mm |
| Weight: | approx. 88 g |
| Operating range: | 0° C – 40° C |

Technical specifications subject to change without notice.

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Humantechnik GHL AG