# MF-1

**D**

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Seite 2

Blitzmodul MF-1

**GB**

**Operating Instructions**

Page 6

Flash Module MF-1

**F**

**Mode d'emploi**

Page 10

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**NL**

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Pagina 14

Flitsmodule MF-1

**I**

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Pagina 18


Modulo flash MF-1

**E**

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Página 22

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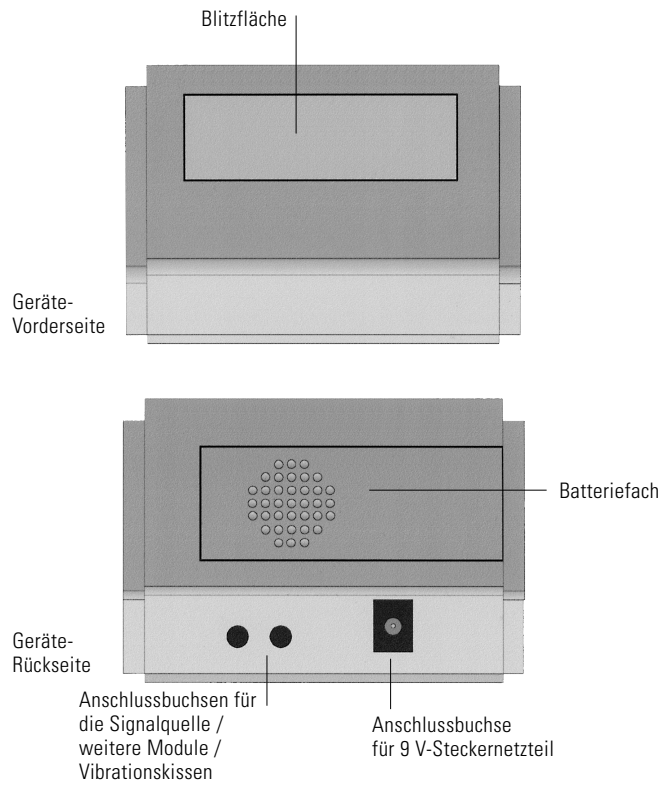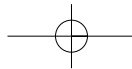
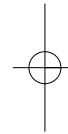
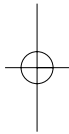
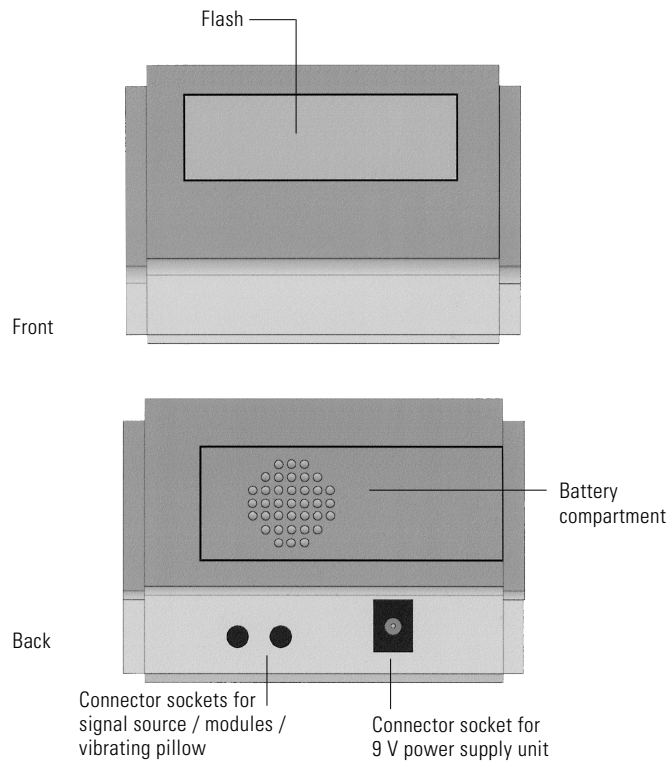
**The *Flash Module MF-1* can be connected to each of our units that has a connector socket (3.5 mm jack) for the vibrating pillow.**

Congratulations on purchasing the *Flash Module MF-1*. You have selected a modern and reliable system. With the *Flash Module MF-1* you can extend every unit that has a connector socket (3.5 mm jack) for the vibrating pillow with a flash signal.

#### **Standard components**

Please check if all listed items are included:

- *Flash Module MF-1*
- Connecting cable
- 9 V-power supply unit
- Warranty card

#### **Installation**

Attach the connecting cable to one of the two connector sockets for the signal source at the back of the *Flash Module MF-1*. Plug the other end of the cable into a unit with a connector socket for a vibrating pillow (e.g. vibrating clock VC-10). Plug the 9 V-power supply unit in the respective socket at the back of the *Flash Module MF-1*.

#### **Note**

A vibrating pillow or another module (e.g. Acoustic Module) can be plugged into the empty connector socket at the back of the *Flash Module MF-1*.

#### **Signalling**

Upcoming signals (e.g. wake-up alarm, lisa alarm) are signalled by very bright flashes.

#### **Special features**

Power failure backup/  
with a 9 V battery  
Battery operation (not included)

The *acoustic module MA-1* can also be operated on just a 9 V battery

### Maintenance and care

The *Flash Module MF-1* does not require any maintenance. If the unit does become dirty, simply wipe it clean with a soft, damp cloth. Never use spirits, thinners or other organic solvents.

Do not set up the unit where it will be exposed to full sunlight for long periods. In addition it must be protected against excessive heat, moisture and severe mechanical shocks.

### Warranty

The *Flash Module MF-1* is a very reliable product. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer.

This warranty covers the repair of the product and returning it to you free of charge. It is essential that you send in the product in its original packaging, so do not throw the packaging away.

The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people who are not authorised to do so (destruction of the seal on the unit).

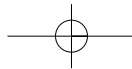
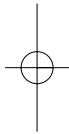
Repairs under warranty are only carried out providing you have filled in and returned the enclosed warranty card from the dealer.

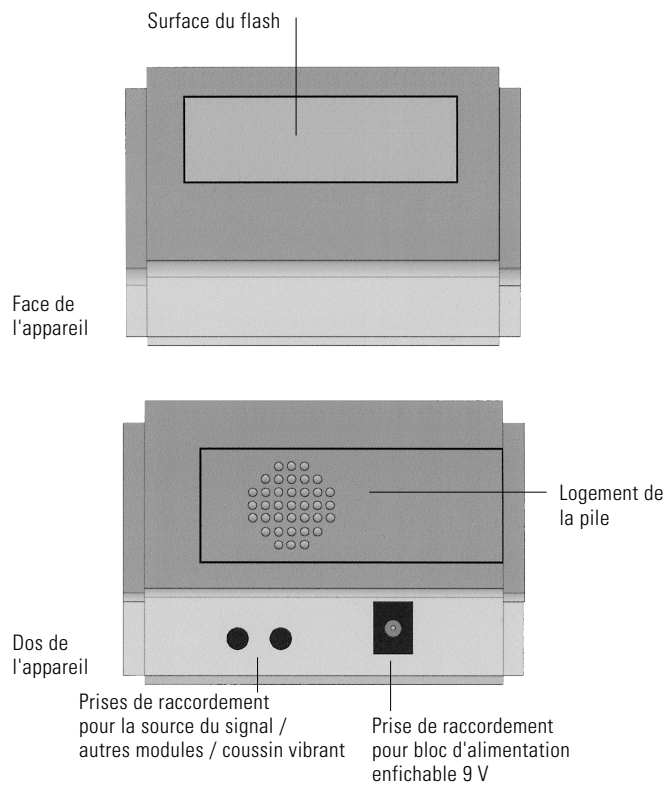
Always specify the product number in any event.

**Technical data**

Power supply: 9 V-power supply unit at 230V, 50H  
or with a 9 V battery

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**Humantechnik GHL AG**