HUMANTECHNIK

PL-100

D

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Seite 2

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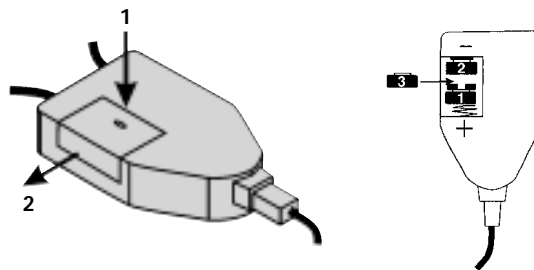
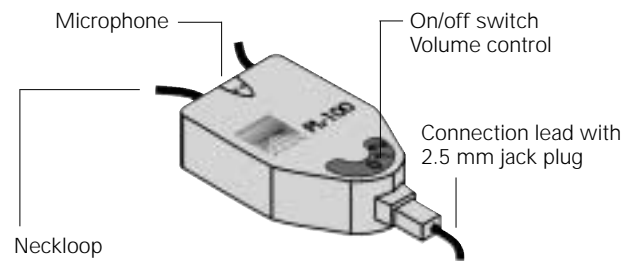
Inserting the batteries

The PL-100 requires three hearing aid batteries (type 675, LR44)

1. Open the cover on the back of the PL-100 by pushing it outwards
2. Insert the three batteries so the + sign matches up with the + mark in the battery compartment (spring contact end).
3. Insert the batteries in the following order:
 - 1 first battery between +ve terminal and plastic stops
 - 2 place the second battery against the -ve terminal
 - 3 insert the third battery between the first and second batteries.
4. Replace the battery cover.

Congratulations on purchasing your *Hands Free Amplifier for Mobile Phones*. We hope it will give you many years of reassuring operation.

Please read through these operating instructions carefully. They tell you how to set up the system correctly and explain all the system's features to you.

Functional principle

The PL-100 generates a loop signal which can be received by a hearing aid with a built-in neckloop. This option is activated by setting the hearing aid to the »T« or »MT« position.

Making and receiving calls

1. Place the neckloop around your neck and plug the 2.5 mm jack plug into the connection jack socket provided on your mobile phone or its adaptor.
2. Switch on the PL-100 and adjust to suitable volume.
3. Use your mobile phone in the normal way (making and receiving calls).
4. Switch off the amplifier by turning the volume control anticlockwise all the way until you hear a click.

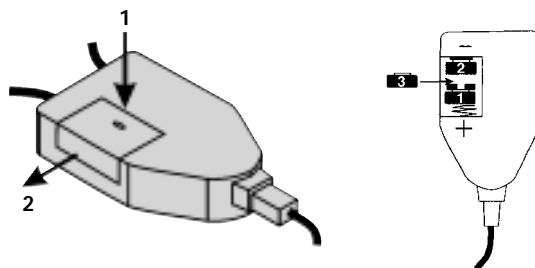
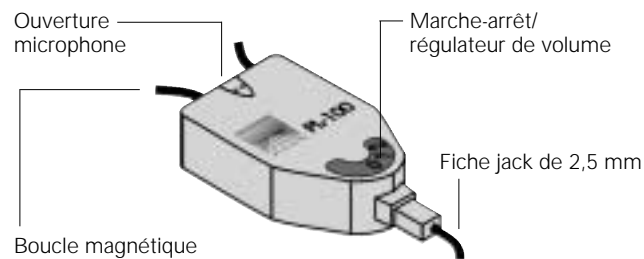
Note

Set the volume control of your mobile phone to maximum so you can hear the voice of the person you are talking to clearly over any background noise.

Technical data

Height: 53 mm
Width: 48 mm
Depth: 26 mm
Colour: grey
Weight: 41 g
Inductive amplification
approx. + 35 dB +/- 10 %

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Humantechnik GHL AG