HUMANTECHNIK

PL-200

D

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Seite 2

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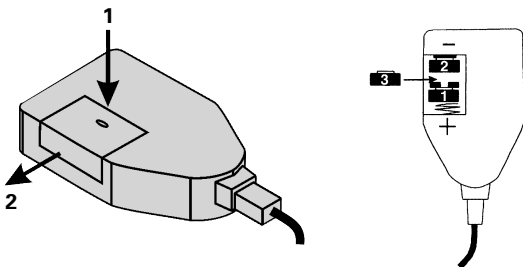
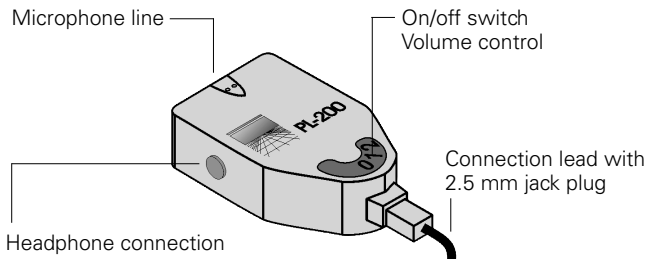
Inserting the batteries

The PL-200 requires three hearing aid batteries (type 675, LR44)

1. Open the cover on the back of the PL-200 by pushing it outwards
2. Insert the three batteries so the + sign matches up with the + mark in the battery compartment (spring contact end).
3. Insert the batteries in the following order:
 - 1 first battery between +ve terminal and plastic stops
 - 2 place the second battery against the -ve terminal
 - 3 insert the third battery between the first and second batteries.
4. Replace the battery cover.

Congratulations on purchasing your *Hands Free Amplifier for Mobile Phones*. We hope it will give you many years of reassuring operation.

Please read through these operating instructions carefully.

They tell you how to set up the system correctly and explain all the system's features to you.

Functional principle

The PL-200 boosts the sound signal of your mobile phone. A 3.5 mm jack can be used for connecting standard headphones (8 to 32 ohm). It is also possible to connect hearing aids directly using an audio connector (monaural/binaural).

Conducting phone calls

1. Connect a set of headphones to the connection socket and insert the 2.5 mm jack plug into the connection jack socket on your handy using a headphone adapter if necessary.
2. Switch on the PL-200 and adjust to suitable volume.
3. Use your mobile phone in the normal way (making and receiving calls).
4. Switch off the amplifier by turning the volume control anticlockwise all the way until you hear a click.

Note

Set the volume control of your mobile phone to maximum so you can hear the voice of the person you are talking to clearly over any background noise.

Maintenance and care

The PL-200 does not require any maintenance. If the unit does become dirty, simply wipe it clean with a soft, damp cloth. Never use spirits, thinners or other organic solvents. Do not set up the unit where it will be exposed to full sunlight for long periods. In addition it must be protected against excessive heat, moisture and severe mechanical shock.

This product is **not** protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, on or near the product.

Warranty

The PL-200 is a very reliable product. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer.

The warranty covers the repair of the product and returning it to you free of charge during the warranty period. It is essential that you send in the product in its original packaging, so do not throw the packaging away. The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorized by Repairs under warranty are only carried out providing you have filled in and returned the enclosed warranty card from the dealer as well as a copy of the sales slip. Always specify the product number in any event.

Disposal

Disposal of used electric and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system).



The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electric and electronic units. You protect the environment and health of your fellow men by the correct disposal of this products. Environment and health are endangered by a faulty disposal. Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of this product from your local community, your communal disposal company or your local dealer.

Technical data

Height:	53 mm
Width:	48 mm
Depth:	26 mm
Colour:	grey
Weight:	41 g
Acoustic amplification	approx. + 25 dB +/- 10 %

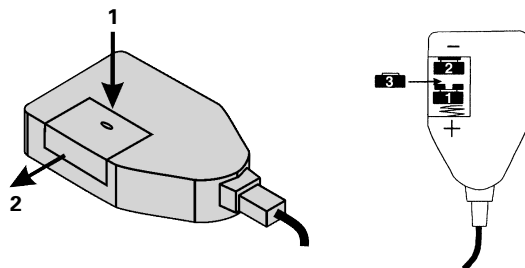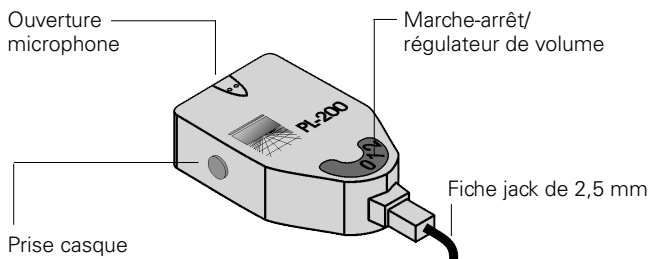Technical specifications subject to change without notice.

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Humantechnik GHL AG