HUMANTECHNIK

PL-51

D

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Seite 2

Telefon-Hörverstärker

GB

Operating Instructions

Page 8

Telephone Amplifier

F

Mode d'emploi

Page 14

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NL

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Pagina 20

Telefoon-hoornverstarker

I

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Pagina 26

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


Operating elements Front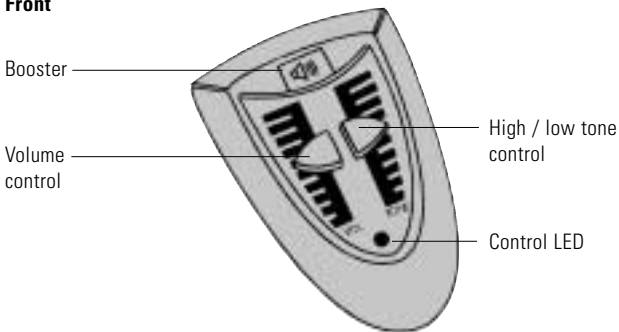

Operating elements Back

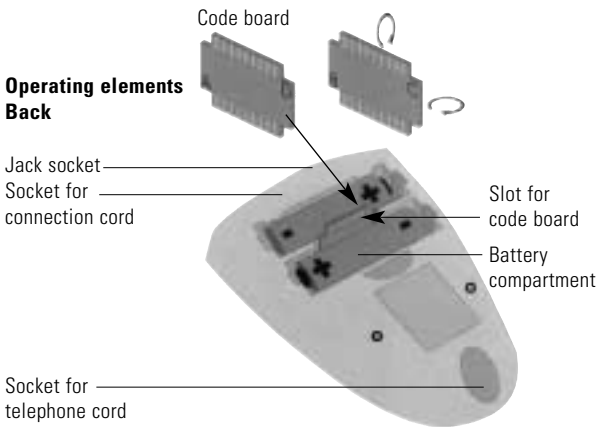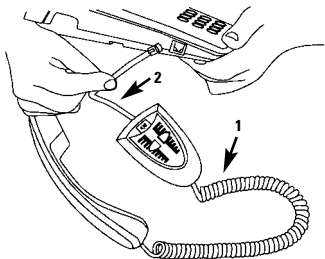Congratulations on purchasing your telephone amplifier *PL-51*. You have selected a modern and reliable system.

Please read the operating instructions thoroughly in order to take the system into operation and to become acquainted with all capabilities of the system.

Start up

Please follow these instructions before using the telephone amplifier:

1. Telephone receiver cord

Unplug the telephone cord from the telephone connection socket. Remove the receiver cord from the cable guide on the underside of your telephone by pushing the locking clip towards the body of the plug and pulling it out together with the cord. This plug of the receiver cord (1) is plugged into the socket on the small side of the *PL-51*. Make sure the plug catches completely.

2. Connection cord

Plug one end of the supplied connection cord (2) into the wide side of the *PL-51* and plug the other end into the receiver socket on the underside of the telephone. Position the cord in the empty cable guide. Finally reconnect the telephone cord to the telephone connection socket. Your telephone is now ready to operate again.

Switch on

The hearing amplifier automatically switches on when the receiver of the telephone is picked up. The green light on top of the unit lights up to indicate that the amplifier is switched on.

Setting the volume / tone

Set the volume control to low before picking up the telephone and adjust the high/low tone control to medium. Now the volume can be adjusted to your needs with the volume control. The high/low tone control can be used to alter the tone by amplifying high and low tones.

Furthermore the output volume can be doubled by pressing the amplifier booster button. The set volume is increased when the amplifier button is pressed down and held. The button must be disengaged while talking.

Switch off

The hearing amplifier switches off automatically approx. 30 seconds after the receiver is placed back on the hook. The green light turns off.

Connection possibilities

The following optional accessories can be connected to the 3.5 mm socket of the *PL-51*:

- Small headphones: e.g. walkman headphones
- Hearing aid users can connect a neck loop to the jack socket. The hearing aids have to be switched to „T“ or „MT“. The volume can be adjusted with the volume control on the *PL-51*.
- Hearing aids with audio shoe can be connected to the *PL-51* with the audio cable monaural (for people wearing one hearing aid) or binaural (for people wearing hearing aids in both ears)

Please note that the loudspeaker of the handset is turned off when the jack socket is used. (Avoidance of feedback).

Accessories

- Neck loop, order no. A-4921-0
- Audio cable monaural, order no. A-4940-0
- Audio cable binaural, order no. A-4950-0
- Stethoscope headset DIR, order no. A-4903-0
- Lightweight stethoscope headset, order no. A-4902-0
- Headphone, order no. A-4905-0
- Earphone A-4904-0

The *PL-51* can be connected to the following units through the jack socket:

- Infrared transmitting systems
- Radio transmitting systems
- Inductive loop systems

Power supply of the hearing amplifier

The *PL-51* is operated with 2 batteries (type AA, LR6), which last up to 2 years. Rechargeable batteries of the same type can also be used.

Please note that when using rechargeable batteries the operating time is shortened. Unplug the telephone cord from the telephone connection socket before changing the batteries. The battery compartment is on the under side of the *PL-51*. Pressing the clip against the cover opens the battery compartment. The cover can now be lifted off. Insert 2 batteries as indicated by the markings in the battery compartment. Then slide the cover back into its original position. Always use 2 new batteries (type AA, LR6) or equivalent rechargeable batteries.

What to do if problems occur

No dial tone when the telephone handset is off the hook / green LED not lit

- Check all plugs for correct connection
- Handset cord on telephone amplifier / receiver
- Connection cord on telephone / telephone connection socket
- If necessary disconnect and reconnect all plugs (faulty contact)

Telephone connection operates but telephone amplifier does not function

- Open the battery compartment and check if the batteries are inserted correctly
- If necessary insert new batteries or fully charged rechargeable batteries

Output volume low

- Batteries are empty, exchange batteries

Adjustment of the *PL-51* – Changing of the wire assignment

As some telephones have different wire assignments the function of the *PL-51* with those telephones is not warranted. However the *PL-51* can be adjusted to different wire assignments. To do this the code board inside the battery compartment has to be replugged.

There are 4 possible positions for the code board: A, B, C and D.

Position A: works with most telephones (delivery status of the *PL-51*)

Position B, C and D: further possible wire assignments

Adjusting different wire assignments:

1. Unplug the telephone cord from the telephone connection socket
2. Open the cover of the battery compartment
3. Take out both batteries
4. Pull the code board out of the slot with caution and turn it.
5. Insert the code board back into the slot.
6. Insert the batteries (mind the polarity).
7. Close the battery compartment and plug the telephone cord back into the telephone connection socket.

Should your telephone amplifier still not work, repeat the above steps.

Maintenance and care

The *PL-51* does not require any maintenance. If the unit does become dirty, simply wipe it clean with a soft, damp cloth. Never use spirits, thinners or other organic solvents. Do not set up the unit where it will be exposed to full sunlight for long periods. In addition it must be protected against excessive heat, moisture and severe mechanical shock.

Warranty

The *PL-51* is a very reliable product. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer. The warranty covers the repair of the product and returning it to you free of charge during the warranty period. It is essential that you send in the pro-

duct in its original packaging, so do not throw the packaging away. The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorized by Repairs under warranty are only carried out providing you have filled in and returned the enclosed warranty card from the dealer as well as a copy of the sales slip. Always specify the product number in any event.

Disposal of used electric and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system).

The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electric and electronic units. You protect the environment and health of your fellow men by the correct disposal of this products. Environment and health are endangered by a faulty disposal. Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of this product from your local community, your communal disposal company or your local dealer.

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Humantechnik GHL AG
ELECTRONIQUE S.A.R.L