HUMANTECHNIK

*Radio Light*

**D**

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Seite 2

**GB**

**Operating Instructions**

Page 10

**F**

**Mode d'emploi**

Page 18

**NL**

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Pagina 26

**I**

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
Pagina 34

**E**

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Página 42

## Anschluss des *RadioLight* Senders

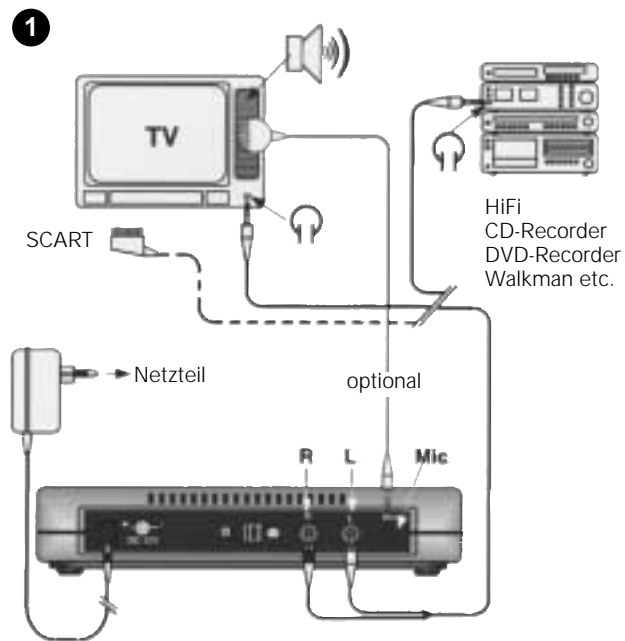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

## Assembling the *RadioLight* transmitter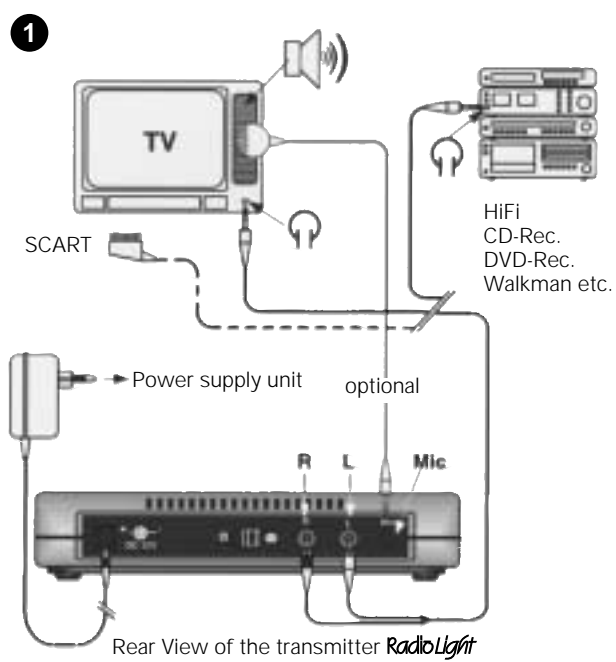


The volume adjustment of the TV, HiFi, CD should be set to a comfortable listening level for other members of the household.

2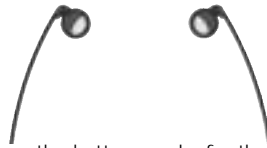

When you first charge the battery packs for the headset receiver let them charge for at least 14 hours. Only use *RadioLight* battery packs.

The *RadioLight* package includes two battery packs and one dummy.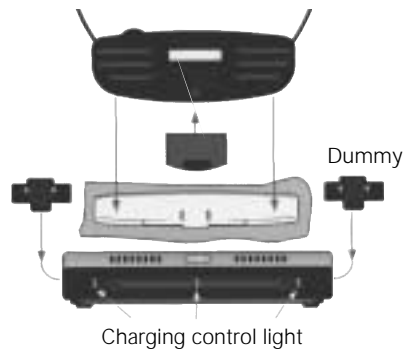

3

### Radio range

The transmitter is designed to have a radio signal range of 80 metres under optimum conditions.

The following conditions may result in a reduced range: (however just moving your position slightly may correct the problem satisfactorily)

- Signal transmission through building structures or vegetation
- Interference from TVs, computer monitors and mobile phones. Under certain circumstances, this may prevent the receiver from functioning at all.


- Positioning of the transmitter at an unfavourable distance from surfaces which reflect radio waves, for example floors and walls. This may lead to the signal wave being weakened or even cancelled out by the reflected wave.
- Metallic objects reduce the range due to the shielding effect of the metal.
- Particularly in towns and cities, there may be many other sources of radio waves which can disrupt the original signal.
- Having units operating at similar frequencies located close to one another may also result in mutual interference.

**4**

#### Channel selector

With the channel selector you can change the frequency of the transmitter. If channel A is distorted you can change to another channel (B or C).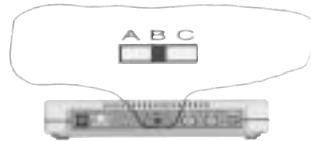


Note: If you change the channel please press the Auto-Tune button of the receiver.

**5**

Auto-tune

Volume adjustment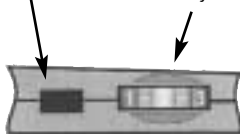


#### Volume adjustment of the RadioLight receiver.

**There is no ON/OFF switch.**

The receiver automatically switches on when worn.

By pressing the Auto-tune button the receiver adjusts itself optimally to the transmitter. The Auto-tune button should be pressed when the sound quality becomes badly distorted.

6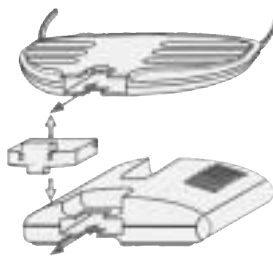

If the sound quality of the receiver becomes noisy and distorted you should replace the battery pack with a charged one or charge the receiver, including battery, in its cradle.

**Warning: do not force the battery pack into the charger or receiver. It fits easily when in the correct orientation.**


7

#### **Cleaning the hygienic covers**

Clean the hygienic covers supplied with the product if they get dirty.

To do this, use the hygiene set (order no. A-4982-0) available as a special accessory.

**8** Please note the following information about your RadioLight:

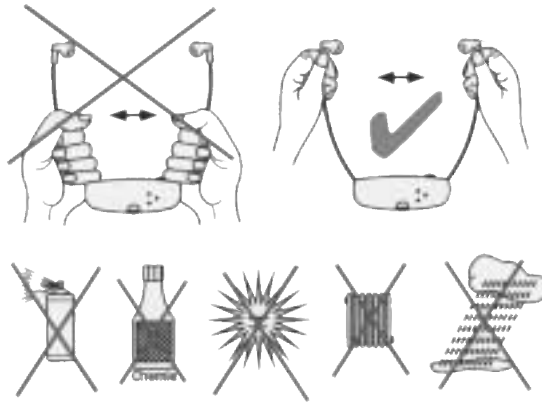If the system needs cleaning ONLY use a soft damp cloth.  
If the foam ear pieces become damaged they should be replaced.

The product warranty does not cover normal wear or damage resulting from misuse, neglect, improper storage, accident, unauthorised repair or alteration.

**9****Technical data:**

Modulation frequency	FM Channel A: 863,5 MHz Channel B: 864,0 MHz Channel C: 864,5 MHz
Frequency response	15 - 20000 Hz
Distortion	< 1 %
Noise level	typ. 60 dB
Battery charging time	approx. 14 h

**Headset receiver:**

Operating time per charge	approx. 6 h
Maximum volume SPL	approx. 120 dBA
Weight	47 g

**Transmitter:**

Power consumption	approx. 3.6 VA
Operating voltage	12 Volt DC
Supply voltage	230 Volt 50 Hz
Weight	approx. 200 g

**10**

Symptom	Cause	Solution
Recharging control lights. Don't light up when batteries are placed into the battery compartments properly.	Transmitter not connected to the power adaptor or no power supply.	Make sure that the power adaptor is connected to the mains power socket and to the transmitter.
Recharging control light does not light up on the same side of the transmitter where a battery is inserted.	No contact has been made between the battery and the recharging contacts of the compartment of the transmitter.	The battery has not been correctly and completely inserted into the battery compartment.
The middle recharger control light does not light up despite having the receiver placed into its compartment in the transmitter.	No contact has been made between the battery in the receiver and the recharging contacts of the compartment in the transmitter.	The receiver has not been properly placed into the compartment. Make sure it is facing in the correct direction.
	There is no battery inserted into the receiver.	Insert a battery into the receiver.
Receiver doesn't work. No audio signals being received.	The transmitter is not correctly connected to the TV or HI-FI audio output. Your TV or HI-FI equipment is not turned on.	Please check and make sure the audio connection cables to the TV or HI-FI are properly and correctly connected. Please read through the owner's manual for your TV or HI-FI equipment to make sure if the output for headphone or line output is connected and it does work correctly.

<b>Symptom</b>	<b>Cause</b>	<b>Solution</b>
Receiver doesn't work. No audio signals being received.	No battery is inserted into the receiver.	Insert a battery into the receiver.
	Battery is discharged	Replace battery with a fully charged one.
The audio transmission is of poor quality, noisy, distorted, unclear or no audio signals being received.	You are too far away with your receiver from the transmitter.	Make sure you are not located too far away from your transmitter.
	If using a 100 Hz television and the RadioLight transmitter is placed on top of it the sound may become distorted in the receiver headset.	Move the RadioLight transmitter to a position where the distortion does not occur.
Battery pack no longer gives the expected operating time per charge.	Battery is discharged	Replace battery with a fully charged one.
	Battery pack is not fully charged.	Recharge the battery for the correct time.
	Battery pack has reached its recharging limit.	Replace battery pack with a new one.

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**Humantechnik GHL AG**