HUMANTECHNIK

RadioLight DIR

D

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Seite 2

GB

Operating Instructions

Page 12

F

Mode d'emploi

Page 22

NL

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Pagina 32

I

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Pagina 42

E

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Página 52

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
Assembling the *RadioLight* DIR transmitter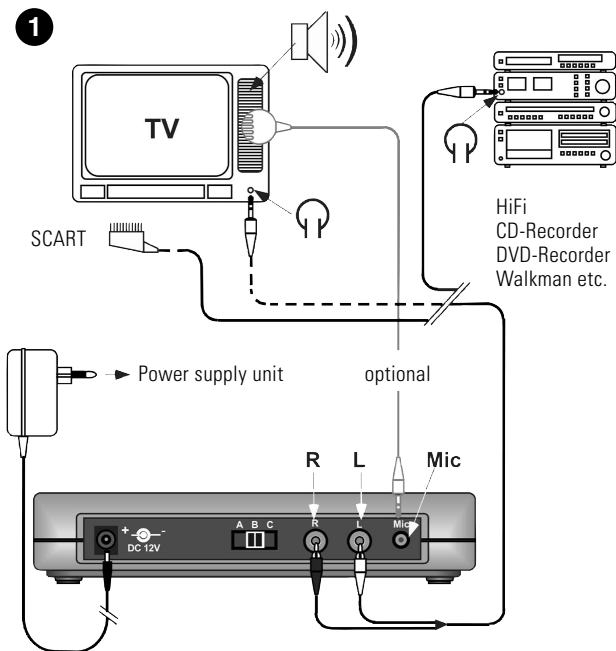


The volume adjustment of the TV, HiFi, CD should be set to a comfortable listening level for other members of the household.

2

When you first charge the battery packs for the headset receiver let them charge for at least 14 hours. Only use original battery packs.

The package includes two battery packs and one dummy.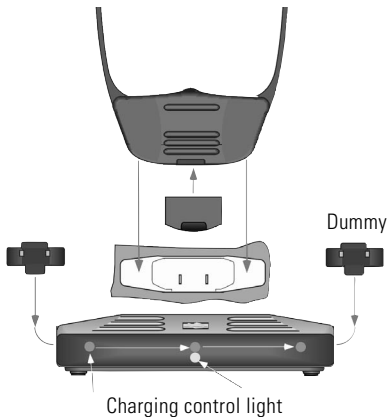

3

Radio range

The transmitter is designed to have a radio signal range of 80 metres under optimum conditions.

The following conditions may result in a reduced range:


- Signal transmission through building structures or vegetation
- Interference from TVs, computer monitors and mobile phones. Under certain circumstances, this may prevent the receiver from functioning at all.

- Positioning of the transmitter at an unfavourable distance from surfaces which reflect radio waves, for example floors and walls. This may lead to the signal wave being weakened or even cancelled out by the reflected wave.
- Metallic objects reduce the range due to the shielding effect of the metal.
- Particularly in towns and cities, there may be many other sources of radio waves which can disrupt the original signal.
- Having units operating at similar frequencies located close to one another may also result in mutual interference.

4


Channel selector

With the channel selector you can change the frequency of the transmitter. If channel A is distorted you can change to another channel.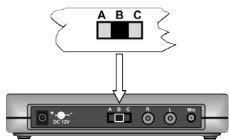


Note: If you change the channel please press the Auto-Tune button of the receiver.

By pressing the **Auto-tune button** (backside of receiver) the receiver adjusts itself optimally to the transmitter. The Auto-tune button should be pressed when the sound quality becomes badly distorted.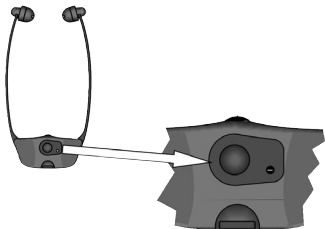

5

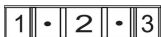Volume adjustment

Volume adjustment of the receiver.

There is no ON/OFF switch.


The receiver automatically switches on when worn.

6

Battery pack replace

If the sound quality of the receiver becomes noisy and distorted you should replace the battery pack with a charged one or charge the receiver, including battery, in its cradle.

Warning: do not force the battery pack into the charger or receiver.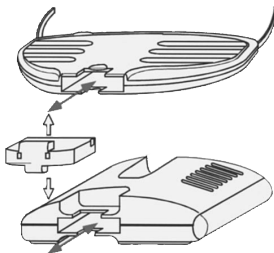

7


Maintenance of the headset receiver

The ear pieces have to be cleaned or exchanged if dirty.

Please use for this the hygienic-set (order no. A-4982-0) which is available as a special accessory.

8

Please note the following informations:

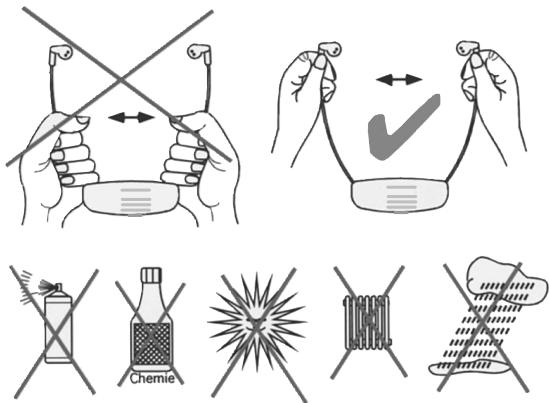If the system needs cleaning **ONLY** use a soft damp cloth.
If the ear pieces become damaged they should be replaced.

The product warranty does not cover normal wear or damage resulting from misuse, neglect, improper storage, accident, unauthorised repair or alteration.

Important Notice

This product is **not** protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, on or near the product.

Technical data:

| | |
|-----------------------|--|
| Modulation frequency | FM Channel A: 863,5 MHz Channel B: 864,0 MHz Channel C: 864,5 MHz |
| Frequency response | 15 - 20000 Hz |
| Distortion | < 1 % |
| Noise level | typ. 60 dB |
| Battery charging time | approx. 14 h |

Headset receiver:

| | |
|---------------------------|-----------------|
| Operating time per charge | approx. 6 h |
| Maximum volume SPL | approx. 120 dBA |
| Weight | 47 g |

Transmitter:

| | |
|-------------------|----------------|
| Power consumption | approx. 3.6 VA |
| Operating voltage | 12 Volt DC |
| Supply voltage | 230 Volt 50 Hz |
| Weight | approx. 200 g |

Technical specifications subject to change without notice.

| Symptom | Cause | Solution |
|--|---|--|
| Recharging control lights. Don't light up when batteries are placed into the battery compartments properly. | Transmitter not connected to the power adapter or no power supply. | Make sure that the power adaptor is connected to the mains power socket and to the transmitter. |
| Recharging control light does not light up on the same side of the transmitter where a battery is inserted. | No contact has been made between the battery and the recharging contacts of the compartment of the transmitter. | The battery has not been correctly and completely inserted into the battery compartment. |
| The middle recharger control light does not light up despite having the receiver placed into its compartment in the transmitter. | No contact has been made between the battery in the receiver and the recharging contacts of the compartment in the transmitter. | The receiver has not been properly placed into the compartment. Make sure it is facing in the correct direction. |
| | There is no battery inserted into the receiver. | Insert a battery into the receiver. |
| Receiver doesn't work. No audio signals being received. | The transmitter is not correctly connected to the TV or HI-FI audio output. | Please check and make sure the audio connection cables to the TV or HI-FI are properly and correctly connected. |
| | Your TV or HI-FI equipment is not turned on. | Please read through the owner's manual for your TV or HI-FI equipment to make sure if the output for headphone or line output is connected and it does work correctly. |

| Symptom | Cause | Solution |
|--|--|--|
| Receiver doesn't work. No audio signals being received. | No battery is inserted into the receiver. | Insert a battery into the receiver. |
| | Battery is discharged | Replace battery with a fully charged one. |
| The audio transmission is of poor quality, noisy, distorted, unclear or no audio signals being received. | You are too far away with your receiver from the transmitter. | Make sure you are not located too far away from your transmitter. |
| | If using a 100 Hz television and the RadioLight transmitter is placed on top of it the sound may become distorted in the receiver headset. | Move the RadioLight transmitter to a position where the distortion does not occur. |
| Battery pack no longer gives the expected operating time per charge. | Battery is discharged | Replace battery with a fully charged one. |
| | Battery pack is not fully charged. | Recharge the battery for the correct time. |
| | Battery pack has reached its recharging limit. | Replace battery pack with a new one. |

Disposal

Disposal of used electric and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system).

The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electric and electronic units.

You protect the environment and health of your fellow men by the correct disposal of this products. Environment and health are endangered by a faulty disposal. Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of this product from your local community, your communal disposal company or your local dealer.

Warranty

This warranty covers the repair of the product and returning it to you free of charge. It is essential that you send in the product in its original packaging, so do not throw the packaging away.

The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people who are not authorised to do so (destruction of the seal on the unit).

Repairs under warranty are only carried out providing you have filled in and returned the enclosed warranty card from the dealer.

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Humantechnik GHL AG