HUMANTECHNIK

**shake
awake®**

D

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Seite 2

Vibrationswecker

GB

Operating Instructions

Page 6

Vibration alarm clock

F

Mode d'emploi

Page 10

Réveil vibrant

NL

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Pagina 14

Trilwekker

I

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Pagina 18

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E

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Página 22

Despertador vibratorio

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Thank you for choosing the *shake awake*. You have purchased a modern and reliable equipment.

Key functions

Selects vibrating alarm

Disable alarm

Selects audible alarm or emits alarm via the special accessories connected

Stop alarm + illuminate display

Snooze alarm

Lock (disables all »SET« buttons)

Set time

Set alarm

Functional principle

The *shake awake* is an electronic clock with a digital time display. It wakes up with an acoustic signal or with vibrating.

Installation

Included in the standard package:

- *shake awake*
- 1 x Micro-battery (type LR03)
- 2 x Mignon-batteries (type LR6)
- Operating instruction
- Warranty card

1. To set time

- Slide set switch to 
Press HOUR/MINUTE to actual time.
- After setting slide switch to  (LOCK).

2. To set alarm

- Slide set switch to 
Press HOUR/MINUTE continuously until desired alarm time appears.
- After setting slide switch to  (LOCK).

3. To active alarm

- Choose vibrating  or audible alarm 
 will appear on display.
- When alarm activates slide alarm switch to  to disable alarm or press  to stop alarm.
 will still appear and alarm will repeat every 24 hours.

4. To active alarm with snooze function ZZZ

- Slide set switch to 
- Press HOUR/MINUTE continuously until desired alarm time appears.
- Slide set switch to ZZZ position. ZZZ will appear on display.
- Choose vibrating  or audible alarm 
- When snooze alarm activates press  to stop.
ZZZ will flash continuously and alarm will repeat every 8 minutes.
- To disable snooze alarm return set switch to  position.
ZZZ will disappear from display.

Note:

To have snooze alarm repeat every 8 minutes you must disable alarm within 1 minute from start. Flashing ZZZ on the display indicates snooze alarm is still activated.

Reminder:

shake awake requires 1 AAA (included) and 2 x AA (not included) batteries. Replace AA batteries when vibrating motor becomes weak. When display becomes dim replace AAA battery.

5. Altering the clock display

You can use the set switch in the battery compartment to switch the clock display between 12-h and 24-h format. To alter the time format, proceed as follows:

- Remove the battery compartment cover.
- Remove the batteries from the battery compartment.
- Slide the set switch to the desired time format and insert the batteries back into the battery compartment.
- Replace the battery compartment cover.

Connection options

The following special accessories can be connected to the *shake awake's* 3.5 mm cinch jack:

- Vibration pillow
- Flash module
- Switch module
- Acoustic module

Before the wake-up alarm can be emitted via a device connected to the cinch jack, the left switch has to be set to  for the audible alarm.

Maintenance and care

The *shake awake* does not require any maintenance. If the unit does become dirty, simply wipe it clean with a soft, damp cloth. Never use spirits, thinners or other organic solvents. Do not set up the unit where it will be exposed to full sunlight for long periods. In addition it must be protected against excessive heat, moisture and severe mechanical shock.

Warranty

The *shake awake* is a very reliable product. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer. The warranty covers the repair of the product and returning it to you free of charge during the warranty period. It is essential that you send in the product in its original packaging, so do not throw the packaging away. The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorized by Repairs under warranty are only carried out providing you have filled in and returned the enclosed warranty card from the dealer as well as a copy of the sales slip. Always specify the product number in any event.

Disposal

Disposal of used electric and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system).

The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electric and electronic units. You protect the environment and health of your fellow men by the correct disposal of this products. Environment and health are endangered by a faulty disposal. Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of this product from your local community, your communal disposal company or your local dealer.

Technical data

Dimensions: 92 x 67 x 30 mm (H x W x D) with base folded in
Weight: 158 g incl. batteries
Power supply: 1x 1.5 V micro battery (AAA – Type LR03) and
2 x 1.5 V mignon batteries (AA – Type LR6) for the
vibration motor

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Humantechnik GHL AG