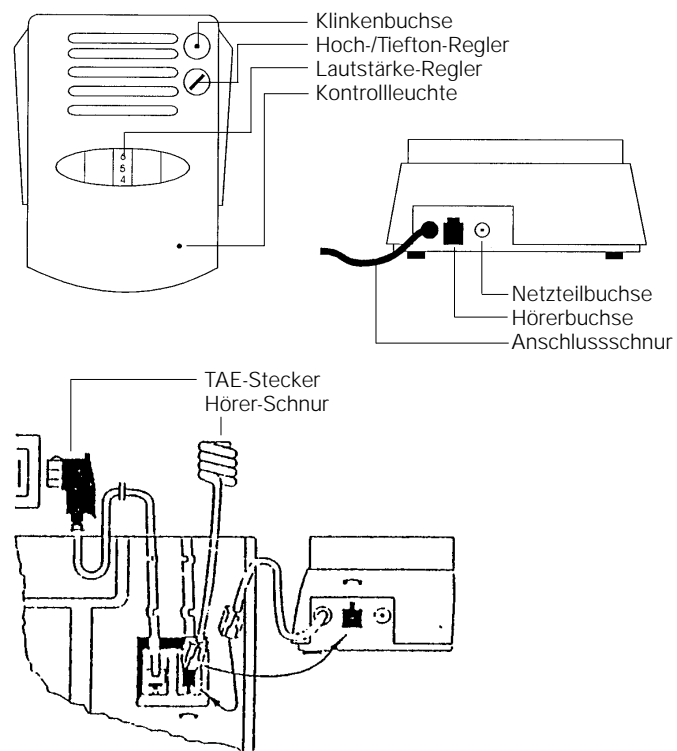
Telefon-Hörverstärker
Telephone Amplifier
Amplificateur téléphonique
Telefoonhoornversterker
Amplificatore telefonico
Amplificador audiométrico para
teléfono

TA-2

Bedienungsanleitung
Operating Instructions
Mode d'emploi
Gebruiksaanwijzing
Istruzioni per l'uso
Instrucciones de servicio

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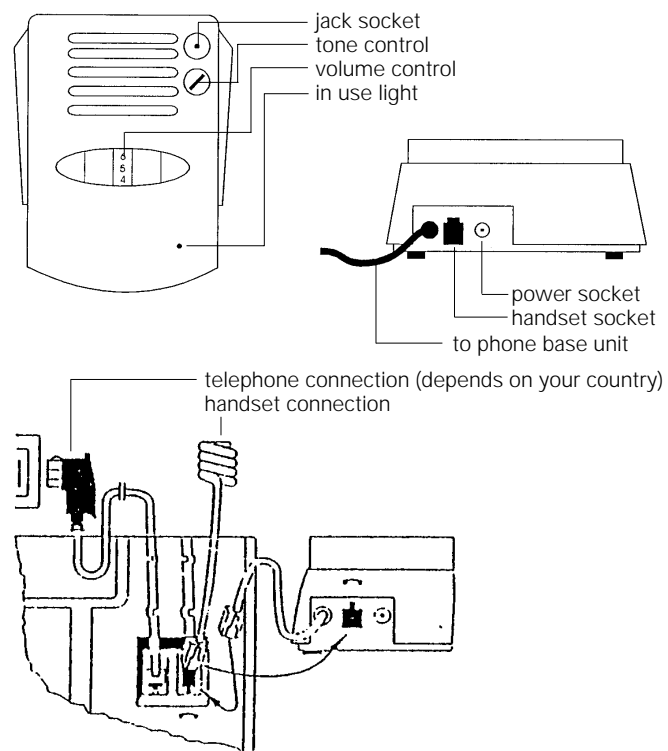
Operating Instructions

Operation

Before using the telephone amplifier please follow these instruction.

1. Telephone receiver cord

First take the wallplug out of the socket for your telephone connection. Remove the receiver cord from its cable guide on the underside of your telephone. To do this, push the locking clip towards the body of the plug at the same time as pulling the plug out of the socket using the cord. Take this plug on the end of the telephone receiver cord and push it into the socket on the back of the telephone amplifier so the plug clips into place. The socket is identified with a telephone receiver signal.

2. Connection cord

Take the plug on the end of the connection cord for the hearing amplifier and push it into the receiver socket on the base of the telephone. Position the cord in the empty cable guide. Finally reconnect the wallplug into the wall socket for your telephone connection. Your telephone is now ready to operate again.

Switching on

The hearing amplifier automatically switches itself on when you pick up the handset of the telephone. The green light on top of the unit lights up to indicate this.

Setting the volume / tone

You can use the tumbler dial on top of the hearing amplifier to adjust the volume of the signal to a comfortable level. Turning the dial towards the larger numbers on the scale increases the volume, and vice versa. You will be able to feel the stop at either end of the dial's travel. Please do not try to force the dial past these points. In addition, you can adjust the tone using the tone control.

Switching off

The hearing amplifier switches itself off automatically about 60 seconds after you put the handset back on the hook. The green light goes out again.

Possible connections

The TA-2 is equipped with a 3.5 mm stereo jack socket to which the following accessories can be connected:

- Mini-headphones, e.g. Walkman headphones
- Hearing aid wearers can connect a neckloop to the jack socket. The hearing aid must be set to »T« or »MT« in this case. It is easy to set the volume using the volume control on the TA-2.
- Hearing aids with a direct audio input allow a direct link to be established between the telephone and the hearing aid. Both a monaural cable (for wearers of a hearing aid in one ear) and a binaural cable (for wearers of hearing aids in both ears) are available for this.
- Infrared transmission systems
- Radio transmission systems
- Induction loop transmission systems

Power supply for the hearing amplifier

Mains operation

Only use the mains adapter supplied. Take the plug on the end of the mains adapter cable and insert it into the socket on the back of the telephone amplifier. This socket is identified by a mains adapter symbol. Then plug the mains adapter into a standard 230 V domestic outlet.

If mains power is not available the unit will not function without batteries.

Battery operation

Open the battery compartment by pushing the battery compartment cover on the underside of the hearing amplifier out to the side. Insert 4 batteries as indicated by the markings in the battery compartment. Then slide the cover back into its original position. Always use 4 new batteries (type AA, LR6) or equivalent rechargeable batteries.

Please note that the loudspeaker in the telephone handset is switched off when a unit is connected to the jack socket (to avoid feedback).

Special accessories

- Audio loop order no. A-4921-0
- Monaural cable order no. A-4940-0
- Binaural cable order no. A-4950-0

The TA-2 can also be connected to the following systems using the jack socket:

What you can do in case of problems

Not every problem means there is a serious fault in your telephone amplifier. Please check the following points before contacting your supplier.

No dial tone when telephone handset off the hook

- Check all the plugs to see whether they are connected properly.
- Handset cord on telephone amplifier.
- Connection cord on unit / telephone socket.
- If necessary, disconnect and then reconnect all the plugs (faulty contact).

Telephone connection is OK, but telephone amplifier does not function

- Check the mains adapter connection.
- Open the battery compartment and check the batteries are inserted correctly.
- If necessary, fit new batteries or equivalent, charged rechargeable batteries.

Telephone amplifier emits loud peeping sounds

- Reduce the volume slightly, and make sure that the tele-

phone handset is not directly adjacent to the hearing amplifier.

- If necessary, fit new batteries or equivalent, charged rechargeable batteries.

Maintenance and care

All units are maintenance-free. If the unit does become dirty, simply wipe it clean with a soft, damp cloth. Never use spirits, thinners or other organic solvents.

Warranty

This warranty covers the repair of the product and returning it to you free of charge.

It is essential that you send in the product in its original packaging, so do not throw the packaging away. The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorised to do so (destruction of the seal on the unit). Repairs under warranty are only carried out providing you have filled in and returned the enclosed warranty card from the dealer or the original warranty certificate if you purchased the unit directly from the manufacturer.

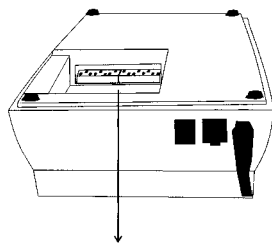
Procedure for adapting the TA-2

Due to the different cable connection of some telephones, the correct function of the TA-2 cannot be guaranteed with these telephones. However, the TA-2 can be adapted to different cable connections by altering the DIP switch settings in the battery compartment of the TA-2.


There are basically two different settings:

Standard setting (the TA-2 is supplied set to this setting)

Setting for variant 2

Standard setting

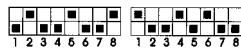Setting for type 2

Important!

1. Please use a small screwdriver when altering the DIP switch settings!

2. Make sure the DIP switches are fully positioned in their end positions.

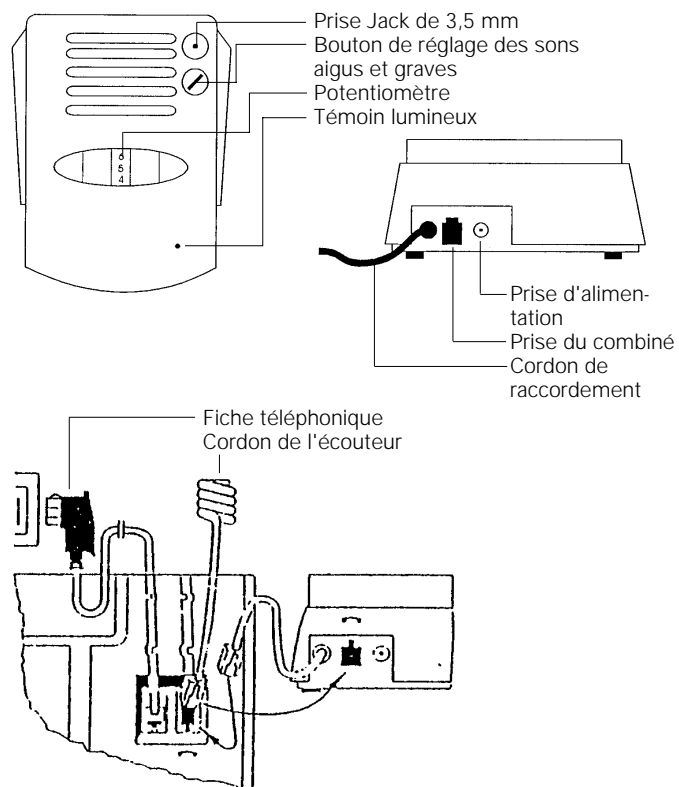
3. Do not set the DIP switches to any other settings.

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Humantechnik GHL AG
ELECTRONIQUE S.A.R.L