HUMANTECHNIK

lisa

D

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Seite 2

Tischblitzlampe

GB

Operating Instructions

Page 6

Table flash lamp

F

Mode d'emploi

Page 10

Lampe-flash de table

NL

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Pagina 14

Tafelflitslamp

I

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Pagina 18




Lampada flash da tavolo

E

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Página 22

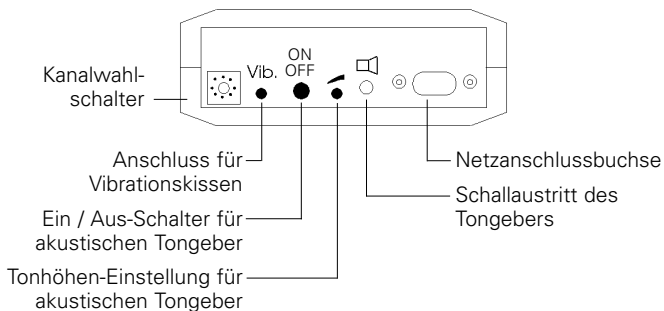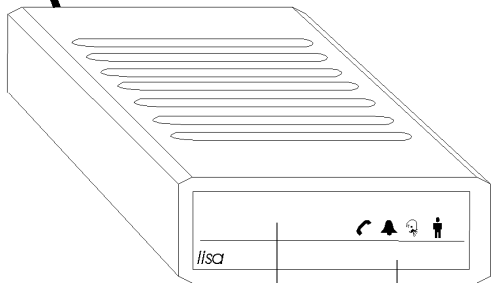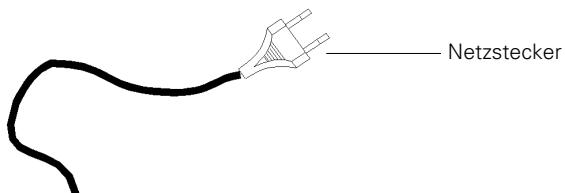Lámpara de destellos de mesa

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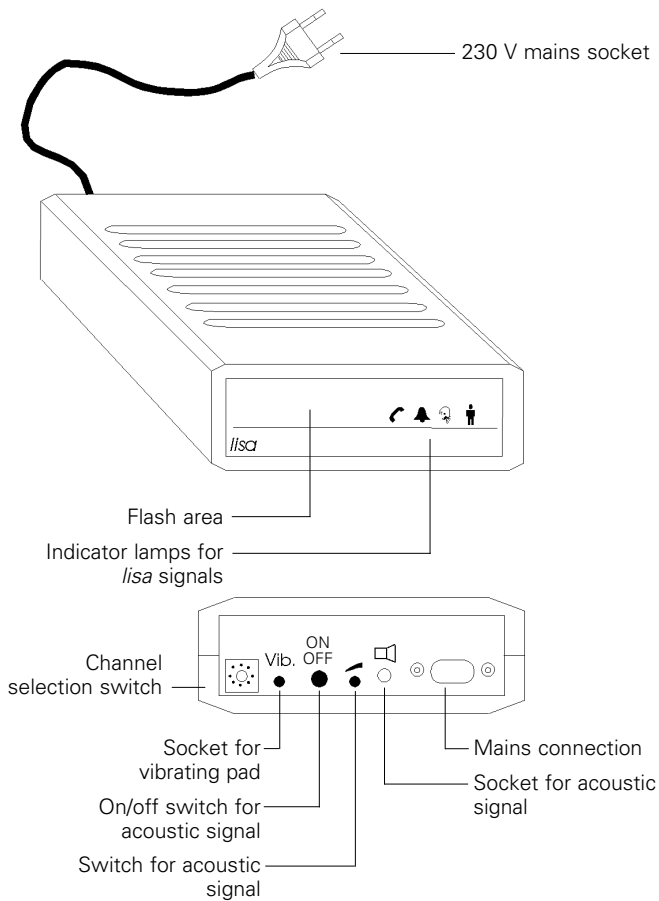
Congratulations on purchasing the *Table Flash Lamp*. We hope it will give you many years of reassuring operation.

Please read through these operating instructions carefully. They tell you how to set up the system correctly and explain all the system's features to you.

lisa system introduction

An alerting system requires at least one transmitter and a receiver. The transmitter captures sound signals and transforms them into digital signals that are passed to a receiver by the existing mains electricity supply. The receiver then transforms these digital signals into light signals.

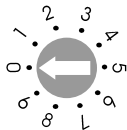
Channel selection

A digital transmission system ensures the correct recognition of each of the digital signals from each transmitter. The system has 10 different selectable channels, 0-9. The factory setting for all lisa devices is channel 0. Only where more than one installation is required in the same building will it be necessary to select a different channel for each system.

To select a channel, turn the rotary switch on the underside of the transmitter to the required channel number using a small screwdriver.

Example:

Set channel number 0

Identification of the flash signals

Each transmitter type produces a different flash sequence. The flash duration is 28 seconds unless it is changed to 8 seconds as described under "Duration of flashing signals" in the transmitter instructions.

Telephone

Telephone is ringing: 2 consecutive flashes for each ring as long as the telephone is ringing. Telephone symbol light illuminates.

Doorbell

Doorbell is pressed: 4 flashes with a short pause - repeated throughout 28 seconds. If 8 second duration is chosen: uninterrupted flashes. Bell symbol light illuminates.

Baby

Baby is crying: 2 flashes followed by a pause – repeated throughout 28 seconds. If 8 second duration is chosen: uninterrupted flashes. Baby symbol light illuminates.

Personal Call Button

Personal call button is pressed on one of the lisa transmitters:

Uninterrupted flashes throughout the 28 or 8 second duration. Person symbol light illuminates.

Alarm

Alarm function is activated: A succession of uninterrupted flashes, clearly different from the other signals, throughout the 28 seconds. If 8 second duration is chosen: uninterrupted flashes. All four symbol lights flash simultaneously.

Symbol lights

The 4 symbol lights help to identify the signals from the different transmitters.

The symbol lights on the receivers remain illuminated for 60 seconds.

Note: During an alarm signal all 4 lights are blinking simultaneously. If doorbell 1 is rung the symbol light will be lit constantly, if doorbell 2 is rung the symbol light will flash.

Important note

All lisa devices connected to the mains supply have a low power consumption which may lead to the device feeling warm. This is normal and does not interfere with the normal function of the devices.

These signalling systems have a high operational safety. Should any faults occur in spite of proper installation and use, please contact your dealer.

High and low voltage spikes (e.g. during a thunderstorm) can occur on the mains supply. The lisa devices have an integrated electronic controller which may stop working after these conditions. If this happens simply unplug each of the devices and plug them in again.

As a precaution remove all devices from their sockets during a thunderstorm.

The use of halogen lamps or other equipment (e.g. Microwave Ovens) may, in rare cases, interfere with the lisa signals. In these cases the use of mains filters or suppressors is advisable. Please ask your dealer.

Maintenance and care

All lisa devices are maintenance free. To clean, remove from their mains sockets and wipe over with a moistened cloth. Never use alcohol, thinner or other organic materials. lisa devices should not be exposed to excessive direct sunlight, heat, thick dust, humidity or mechanical shock.

Note: This product is **not** protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, on or near the product.

Environmental note/Waste disposal

Disposal of used electrical and electronic equipment (applicable in the countries of the European Union and in other European countries which operate a separate collection system for these devices).

The symbol on the product or its packaging indicates that this product should not be treated as normal household waste. It should be handed in to a collection center for recycling electrical and electronic equipment.

By playing your part in ensuring the correct disposal of this product, you will be safeguarding the environment and the health of other people. The environment and health are jeopardized by incorrect disposal.

Material recycling helps to reduce the consumption of raw materials. You can obtain further information about recycling this product from your local council, from local waste disposal operators and from the shop where you bought the product.

Warranty

This warranty covers the repair of the product and returning it to you free of charge. It is essential that you send in the product in its original packaging, so do not throw the packaging away.

The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people who are not authorised to do so (destruction of the seal on the unit).

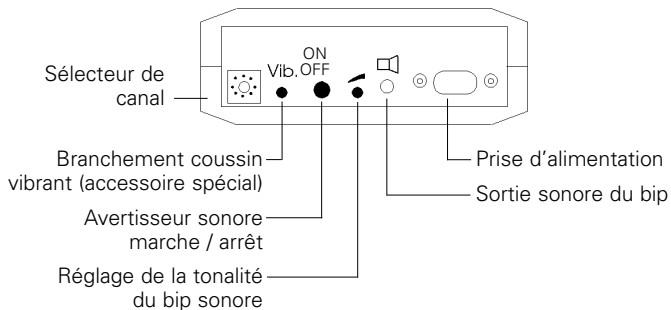
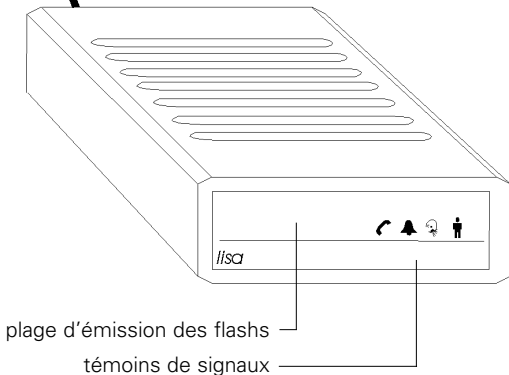
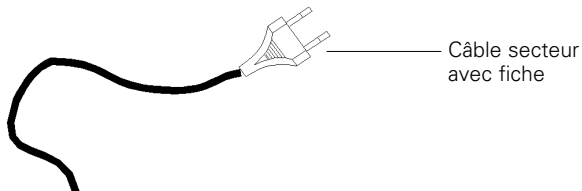
Repairs under warranty are only carried out providing you have filled in and returned the enclosed warranty card from the dealer.

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Humantechnik GHL AG
ELECTRONIQUE S.A.R.L